

Signia Pocket Guide

Fall 2023 – Introducing Signia Integrated Xperience with
RealTime Conversation Enhancement

veterans.signiausa.com

Integrated Xperience (IX),
Augmented Xperience (AX),
and Xperience (X) Platforms

NEW



Pure Charge&Go T IX
(RIC)



Insio Charge&Go AX
(Custom)



Pure 312 AX
(RIC)



Styletto AX
(RIC)



Motion Charge&Go X (BTE)
Standard, Power and Super Power



signia

Expanded Portfolio Offerings

NEW



Pure
Charge&Go
TIX

NEW



CROS Pure
Charge&Go
IX



Motion
Charge&Go
X



Motion
Charge&Go P
X



Motion
Charge&Go SP
X



Insio
Charge&Go
AX



Pure 312
AX



CROS Pure 312
AX



Styletto
AX



CROS Styletto
AX

Signia is dedicated to improving the quality of life for all veterans, active military, and American Indians through better hearing. Through innovative solutions, unparalleled support, and service excellence, Signia is proud to work with Government Services audiologists to enhance their patients' human performance and help them experience the world around them on their terms.

Introducing Signia Integrated Xperience with RealTime Conversation Enhancement. The world's first hearing aid that can identify, enhance, and track the location of multiple conversation partners in real time, even as they move. 95% of wearers have better speech reception with RealTime Conversation Enhancement than without.*

In addition to these new products, Signia reaffirms its commitment to Government Services audiologists and their patients through its Signia Delivers promise. Sharing their goal of achieving the best patient outcomes possible, Signia Delivers across three pillars:

- **Innovative Solutions:** In addition to its robust product offerings, including the world's largest portfolio of rechargeable solutions, Signia has introduced pioneering solutions like Signia Remote Care for virtual hearing care and adjustments and Face Mask Mode.
- **Unparalleled Support:** The company's recently expanded team of audiology, customer care, and tech support professionals brings an unrivaled 180 combined years of Government Services expertise to understand audiologists' needs and assist with product selection, challenging fittings, personalized training, and more.
- **Service Excellence:** Signia strives to handle each order as efficiently and professionally as possible, maintaining a laser focus on enhancing order fulfillment and reducing delivery time.

*Visit signia-pro.com/en-us/claims-data for more details.

Table of Contents

Our People	5
------------------	---

Audiology

Signia Integrated Xperience	11
Signia Augmented Xperience	12
Signia Signature Features	13
Signia Signature Audiological Features	14
Be Active, Be Healthy, Be Brilliant with My WellBeing.	16
Signia Xperience Platform	17
Signia App for Patients	18
Signia TeleCare – 3.0	19
Connexx 9.11.0 UPDATE	20

Technology

Pure Charge&Go T IX	21
Insio Charge&Go AX Customs	22
Pure Charge&Go T AX	24
Pure 312 AX	25
Styletto AX	26
CROS — RICs	27
Signia RIC products and fitting ranges	28
miniReceiver 3.0	29
EarWear 3.0	30
Motion Charge&Go X	31
Thin Tube 3.0	33
Insio Nx Customs	34
Motion 13 and 13 P Nx	36

Streamers

StreamLine™ Mic	37
StreamLine™ TV	40

Remote Controls

miniPocket™	43
Pairing hearing aids with miniPocket	44

Chargers

Rechargeability	45
Portable IX/AX Charger	46
Insio Charge&Go AX Charger	47
Closed-lid Inductive Charger	48
Styletto AX Charger	49
Dry&Clean Charger	50
How to order a Charger in ROES	52

Earmolds	53
----------------	----

Signia Quick Reference

Signia Nomenclature	55
Where to Find Serial Numbers	56
Charger Compatibility	57
How to Measure for Signia 3.0 Receivers	59
Signia Earwear 3.0 Parts	60
Signia Wax Guard Guide	61
Earmold Ordering Tips	62
Accessories Tips	63
Signia Fitting Guide	64
Tinnitus Fitting Guide	67
CROS Pure Charge&Go T IX, AX, X & CROS Pure 312 AX Quick Fit Guide ..	68
CROS/BICROS Listening Check	69
How to Pair	70

ROES Codes	71
------------------	----

YOUR SIGNIA GOVERNMENT SERVICES TEAM

Phone: (800) 955-4327

Fax: (732) 562-6653

Consumer Hotline: (800) 350-6093

Website: veterans.signiausa.com

Hours of Support: 7am - 8 pm EST

WEST



Government Account Manager
Caitlin Turilli, Au.D.
(206) 761-8081
caitlin.turilli@signiausa.com



Government Audiologist
Hannah Williams, Au.D.
hannah.williams@signiausa.com



Customer Care
Daniela Valencia
daniela.valencia@signiausa.com



MOUNTAIN



Government Account Manager
Suman Burua, Au.D.
(385) 347-6469
suman.burua@signiausa.com



Government Audiologist
Crystal Bles Dvorak, Au.D.
crystal.dvorak@signiausa.com



Customer Care
Daniela Valencia
daniela.valencia@signiausa.com

CENTRAL



Government Account Manager
Peggy Zanniller, BS Speech & Hearing Science
(952) 221-8279
margaret.zanniller@signiausa.com



Government Audiologist
JoAnn Kerwin, Au.D.
joann.kerwin@signiausa.com



Customer Care
Kenya Barley
kenya.barley@signiausa.com

SOUTH



Government Account Manager
Janelle Scanlon, M.A.
(913) 291-4570
janelle.scanlon@signiausa.com



Government Audiologist
Alaina Feliz, Au.D.
alaina.feliz@signiausa.com



Customer Care
Kenya Barley
kenya.barley@signiausa.com

NORTHEAST



Government Account Manager
Kristen Lewis, M.S.
(724) 799-9401
kristen.lewis@signiausa.com



Government Audiologist
Rachel Pokorney, Au.D.
rachel.pokorney@signiausa.com



Customer Care
Sarah Andresen
sarah.andresen@signiausa.com

MID ATLANTIC/PUERTO RICO



Government Account Manager
Lindsay Roberts, Au.D.
(724) 812-8769
lindsay.roberts@signiausa.com



Government Audiologist
Stephanie Porowski, Au.D.
stephanie.porowski@signiausa.com



Customer Care
Sarah Andresen
sarah.andresen@signiausa.com

SOUTHEAST



Government Account Manager
Spencer Reeder, Au.D.
(352) 449-9659
spencer.reeder@signiausa.com



Government Audiologist
Courtney Rogers, Au.D.
courtney.rogers@signiausa.com



Customer Care
Sarah Andresen
sarah.andresen@signiausa.com

GS MANAGEMENT TEAM



VP Government Services
John Muray
john.muray@signiausa.com



Government Services Director of Sales
Brian Fasler, Au.D.
brian.fasler@signiausa.com



Government Services Manager
Kerin McDevitt
kerin.mcdevitt@signiausa.com



Government Services Lead Engineer
Andrei Bucur
andrei.bucur@signiausa.com



Government Services Director of Product Management
Erin Robison
erin.robison@vsa.com



Government Services Operations Sr Supervisor
Rajal Patel

GOVERNMENT SERVICES SUPPORT



Government Services Customer Care
Rajesh Khatri

Your Signia Team

Phone: (800) 955-4327 • Fax: (732) 562-6653

Consumer Hotline: (800) 350-6093

Northeast Team



Government Account Manager

Kristen Lewis, M.S.

(724) 799-9401

kristen.lewis@signiausa.com



Government Audiologist

Rachel Pokorney, Au.D.

rachel.pokorney@signiausa.com



Customer Care

Sarah Andresen

sarah.andresen@signiausa.com

Southeast Team



Government Account Manager

Spencer Reeder, Au.D.

(352) 449-9659

spencer.reeder@signiausa.com



Government Audiologist

Courtney Rogers, Au.D.

courtney.rogers@signiausa.com



Customer Care

Sarah Andresen

sarah.andresen@signiausa.com

Your Signia Team

Phone: (800) 955-4327 • Fax: (732) 562-6653

Consumer Hotline: (800) 350-6093

South Team



Government Account Manager

Janelle Scanlon, M.A.
(913) 291-4570
janelle.scanlon@signiausa.com



Government Audiologist

Alaina Feliz Au.D.
alaina.feliz@signiausa.com



Customer Care

Kenya Barley
kenya.barley@signiausa.com

Central Team



Government Account Manager

Peggy Zanmiller, B.S. Speech and Hearing Science
(952) 221-8379
margaret.zanmiller@signiausa.com



Government Audiologist

JoAnn Kerwin, Au.D.
joann.kerwin@signiausa.com



Customer Care

Kenya Barley
kenya.barley@signiausa.com

Your Signia Team

Phone: (800) 955-4327 • Fax: (732) 562-6653

Consumer Hotline: (800) 350-6093

Mountain Team



Government Account Manager

Suman Burua, Au.D.

(385) 347-6469

suman.burua@signiausa.com



Government Audiologist

Crystal Dvorak, Au.D.

crystal.dvorak@signiausa.com



Customer Care

Daniela Valencia

daniela.valencia@signiausa.com

West Team



Government Account Manager

Caitlin Turriff, Au.D.

(206) 761-8081

caitlin.turriff@signiausa.com



Government Audiologist

Hannah Williams, Au.D.

hannah.williams@signiausa.com



Customer Care

Daniela Valencia

daniela.valencia@signiausa.com

Your Signia Team

Phone: (800) 955-4327 • **Fax:** (732) 562-6653

Consumer Hotline: (800) 350-6093

Mid Atlantic Team



Government Account Manager

Lindsay Roberts, Au.D.

(724) 812-8769

lindsay.roberts@signiausa.com



Government Audiologist

Stephanie Porowski Au.D

Stephanie.porowski@signiausa.com



Customer Care

Sarah Andresen

sarah.andresen@signiausa.com



Your Signia Team

Phone: (800) 955-4327 • **Fax:** (732) 562-6653

Consumer Hotline: (800) 350-6093

Management Team



VP Government Services
John Murray
john.murray@signiausa.com



Director of Sales
Brian Fasler
brian.fasler@signiausa.com



Manager of Government Services
Kerrin McDevitt
kerrin.mcdevitt@signiausa.com



Lead Engineer
Andrei Bucur
andrei.bucur@signiausa.com



Director of Product Management
Erin Robison
erin.robison@wsa.com



Government Services Operations Sr Supervisor
Rajal Patel

Signia Integrated Xperience

Finally, a hearing aid designed for real conversations

Meet the world's first hearing aid with a multi-stream architecture capable of accurately pinpointing multiple moving speakers in real-time conversation.

Improved hearing is about more than just sound amplification. It is about empowering individuals to immerse themselves in their surroundings, create meaningful connections, and cherish every moment of life's experiences.

Leveraging our profound understanding of dynamic conversations, including the head movements of the wearer and changing positions of speakers, we crafted Signia Integrated Xperience. This groundbreaking technology ensures uninterrupted conversation engagement in noise, addressing a critical challenge many hearing aid wearers face.

Our commitment is to support wearers in leading active lives, unshackled from the limitations of traditional hearing aids and free to contribute and foster deeper connections.

Experience the transformative power of real-time, dynamic conversation enhancement with Signia Integrated Xperience.



Augmented Xperience Platform at a Glance

The groundbreaking Signia Augmented Xperience (AX) platform is the first hearing aid technology to use split processing. This means that speech and background noise are processed separately and then recombined to deliver outstanding speech clarity – even in noisy environments.



2x

Audiological core
processors

110%

Increased binaural
audio transfer rate

60%

Reduced binaural
latency

27%

More electric
circuits

22%

Longer battery
runtime

Signia Signature Features



Auto EchoShield: Automatically better speech clarity and comfort even in reverberant situations

Automatically analyzes the listening environment and adjusts to overcome problematic reverberation in every kind of room. Wearers will enjoy better speech clarity, less listening effort, better room perception, higher overall satisfaction, higher listening comfort, and better perceived speech intelligibility.

Visit signia-pro.com/en-us/claims-data for more details



Own Voice Processing (OVP) 2.0: Unique solution to increase own voice comfort

Can separate the wearer's own voice from other speech and noise for a natural sound, increasing first-fit acceptance to 88% and keeping patients engaged in conversations.

*Not available with Inzio Charge&Go AX.



HandsFree with CallControl (iOS): Turn hearing aids into a headset

The HandsFree with CallControl (iOS) features let the hearing aid wearer use their Signia IX and AX hearing aids for hands-free calls with their iPhone® in different environments — even in noisy ones. Wearers will enjoy answering the phone in any situation and never being limited in their active, busy lifestyle.

Signia Signature Audiological Features

Enhanced Audiological Features



e2e Wireless 4.0: A great leap for true binaural hearing

e2e 4.0 significantly boosts the speed of data transmission and a 60% reduction in latency provides inaudible shifts between environments. The synchronization of more binaural audio data than ever before ensures clearer speech processing with high-precision directional awareness.



TruEar 360: Mimics the natural pinna effect

The powerful processing engine allows for an exceptional TruEar 360 mic mode that reproduces and emphasizes the capabilities of the natural pinna-effect better than ever. TruEar 360 highlights the difference between front and back sources of sound to make localization of sound sources easier.



Automatic Situation Detection : automatically responds to where and what you are doing

Provides detailed information about the wearer's movement and environment. Analyzing and adapting to the acoustic and motion changes, these sensors deliver a clear understanding of speech from any direction and maximizing audibility as you move throughout the day.

Signia Signature Audiological Features

Enhanced Audiological Features



Enhanced feedback cancellation: No distractions for a smooth sound impression

e2e 4.0 also synchronizes feedback cancellation between the ears in the same phase, removing more artifacts for a smoother overall sound experience.



Direct Streaming: hassle-free listening to audio input

Signia Integrated Xperience and Augmented Xperience offer direct streaming for both iOS and ASHA-compatible Android™ smartphones. They handle the signal of a streaming input independently from other signals, ensuring that the sound is even clearer and easier to optimize.



Increased input dynamic range input: appreciated by music lovers

The microphones and the state-of-the-art analog input stage can handle signals up to 117dB with great sound quality. That's very close to the full capacity of human ears and ensures a clear, crisp signal, for scenarios like concerts.

Be Active, Be Healthy, Be Brilliant with My WellBeing

With the new My WellBeing features in the Signia app, you can now see important health indicators empowering you to better maintain your physical activity.

Research shows that health trackers like My WellBeing help you stay active,* which boosts your energy levels and confidence.**

Get started today with Signia!



My Steps

Staying physically active is shown to improve energy levels and confidence – and even slow down age-related hearing loss.** The motion sensor in Signia AX hearing aids not only helps you hear better on the move, but also enables you to see exactly how many steps you take each day.



My Activity

The use of a fitness tracker can help increase physical activity and keep you active.* Signia My WellBeing is smart enough to know how much time you spend exercising — and together with your steps, provides an overview of your physical activity level.



My WearTime

You can only take advantage of the many benefits your hearing aids provide when you are wearing them. My WellBeing monitors how well you are using your hearing aids based on how often you use them – helping you better understand your usage habits.



My Conversations

Your social interactions play an important role in maintaining good hearing health. Signia's unique Own Voice Processing technology measures how much you are socially interacting with others, so you can know the duration and dynamics of social interactions.

My Conversations only available with OVP 2.0

* Effectiveness of physical activity monitors in adults: systematic review and meta-analysis. BMJ, 376, e068047.

** Fitness and Better Hearing, Part 1. Hearing Review, 21(3): 18-24

Signia Xperience Platform

Signia Xperience focuses on the lifestyle and actions of the wearer for optimized support in every situation. Intuitive and flexible programming software makes Signia Xperience adaptable and user-friendly, just another way we're putting the wearer first.



Acoustic Sensors

Analyzing and adapting to the acoustic changes around you, these sensors deliver personalized amplification and a clear understanding of speech from any direction, mirroring optimized natural hearing experiences.¹

Motion Sensors

Optimizing hearing in every situation by recognizing changes in your own movement to adjust acoustic settings and maximizing audibility as you move throughout the day.²



Own Voice Processing (OVP™)

Separates the wearers own voice from other speech and noise for a natural sound, increasing first-fit acceptance to 88% and keeping patients engaged in conversations.³



Dynamic Soundscape Processing

Utilizing Signia's iconic binaural audio processing with Binaural Directionality, this world's first technology has been clinically proven to provide better than normal hearing in noisy situations.⁴

1 Haubold J. (2019) Real-world effectiveness of Signia Xperience Dynamic Soundscape Processing. Unpublished Manuscript.

2 Froehlich M, Branda E, Freels K. (2019) New dimensions in automatic steering for hearing aids: Clinical and real-world findings. Hearing Review. 26(11):32-36.

3 Høydal, E. H. (2017). A new own voice processing system for optimizing communication. Hearing Review, 24(11), 20-22. Further details: www.signia-pro.com/ovp-study.

4 Froehlich M, Freels K, Powers T. (2015). Speech recognition benefit obtained from binaural beamforming hearing aids: comparison to omnidirectional and individuals with normal hearing AudiologyOnline, Article 14338. Retrieved from <http://www.audiologyonline.com>

Signia App for Patients

Beyond streaming and remote control capabilities, the Signia app is customized to display specific features based on the patient's hearing aid model and gives them freedom and support with access to:




TeleCare™ for remote care support with you! Patients fitted with Signia TeleCare have a 19% higher acceptance rate than patients fitted without it.¹



Signia Assistant for on-demand AI support. 93% stated that using the Signia Assistant improved their satisfaction for listening in difficult situations.²



Signia Face Mask Mode for an immediate boost in speech with the simple tap of a button. Facemasks have been shown to reduce audibility by up to 30%, and make lip reading impossible.³

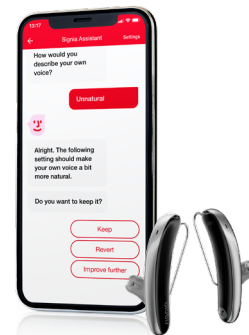

¹ Article, Matthias Froehlich, PhD, Eric Branda, AuD PhD, Dirk Apel, BS, 2020: User Engagement with Signia TeleCare: A Way to Facilitate Hearing Aid Acceptance

² Hoydal EH, Fisher R-L, Wolf V, Branda E, Aubreville M. Empowering the wearer: AI-based Signia Assistant allows individualized hearing care. Hearing Review. 2020;27(7):22-26.

³ Goldin A, Weinstein BE, Shiman N. How do medical masks degrade speech perception? Hearing Review. 2020;27(5):8-9.

Signia TeleCare – 3.0

Proven remote care support to remain virtually connected to your patients

Remain connected with patients — Communicate with patients through Signia's safe and secure connection on any laptop, smartphone, or computer.


Higher patient satisfaction — More flexibility to support patient needs.

- In-person or remote activation of TeleCare for new and existing patients.
- Remote adjustments via Connexx.
- Compatible with multiple Signia platforms (Nx, X, AX, IX).
- Adjustments are sent to the patient in real-time (synchronously).



Patients fitted with Signia TeleCare have a 19% higher acceptance rate than patients fitted without it.*

TeleCare Code
no longer needed for
easier Telecare Use

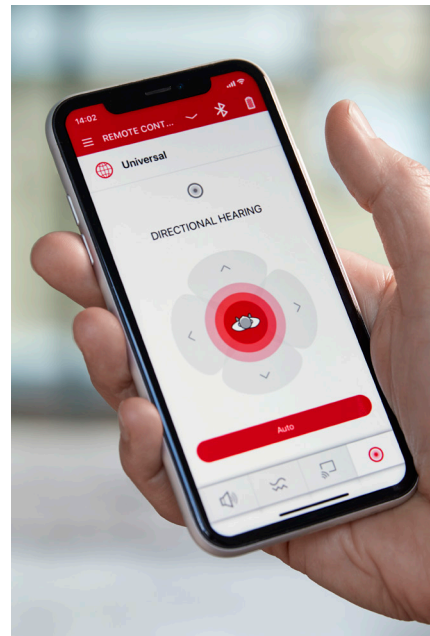

* Article, Matthias Froehlich, PhD, Eric Branda, AuD, PhD, Dirk Apel, BS, 2020: User Engagement with Signia TeleCare: A Way to Facilitate Hearing Aid Acceptance

Connexx 9.11.0 UPDATE

- Improved performance speed of software.
- Wireless hearing aid firmware updates are available via ConnexxAir wireless, only for non-Bluetooth® hearing aids. Firmware updates are only available when received from the notification center.



version **9.11.0** with SIFIT™ 9.11.0

Pure Charge&Go T IX

Unleash the Power of Conversation




36 hours on a single charge
Portable Charge&Go Charger holds 3 charges

- Receivers: 3.0 S, M, P, HP
- EarWear: EarWear 3.0



Key features & benefits

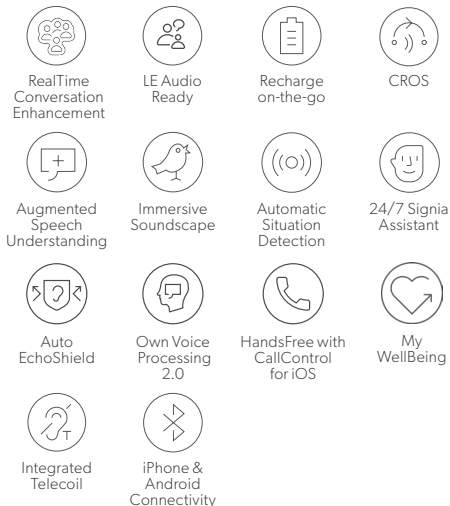

Insio Charge&Go AX Customs



Available in Full Shell, Half Shell, and Canal



Insio Charge&Go AX Charger
(Standard) – features contactless
charging and 24 hours on a
single charge



Black



Dark Brown



Brown



Mocha



Tan



Beige

Key features & benefits



Rechargeable



iPhone
& Android
Connectivity



Immersive
Soundscape



Tinnitus
Relief



Augmented
Speech
Understanding



24/7 Signia
Assistant



Automatic
Situation
Detection



Signia
App



Auto
EchoShield






HandsFree with
CallControl
for iOS



My
WellBeing

Please note that red, blue and
clear shells are also available.

Insio Charge&Go AX Customs

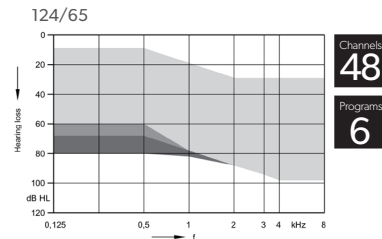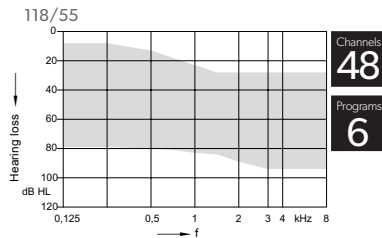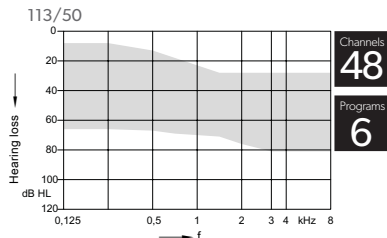

Insio Indicators Available



Large R/L



Large Red/Blue/White Dot



Last Name



Canal Grips

Insio Removal Options



Filament Loop



Removal Post



OtoRing

Pure Charge&Go T AX

The ultimate rechargeable RIC.



36 hours on a single charge
Portable Charge&Go Charger holds 3 charges



Key features & benefits



Integrated
Telecoil



iPhone &
Android
Connectivity



Recharge
on-the-go



CROS



Augmented
Speech
Understanding



Immersive
Soundscape



Automatic
Situation
Detection



24/7 Signia
Assistant



Auto
EchoShield



Own Voice
Processing
2.0



HandsFree with
CallControl
for iOS



My
WellBeing

- Receivers: 3.0 S, M, P, HP
- EarWear: EarWear 3.0

Pure 312 AX



IP68

Key features & benefits



312 Battery



iPhone &
Android
Connectivity



24/7 Signia
Assistant



CROS



Augmented
Speech
Understanding



Immersive
Soundscape



Automatic
Situation
Detection



Auto
EchoShield



Own Voice
Processing
2.0



HandsFree with
CallControl
for iOS



My
WellBeing

- Receivers: miniReceivers 3.0 S, M, P, HP.
- Receivers are field replaceable
- EarWear: EarWear 3.0



Black



Graphite



Dark
Champagne



Silver
Hero



Pearl
White



Fine
Gold



Deep
Brown



Sandy
Brown



Rose
Gold



Beige

Styletto AX

Exchangeable receivers, rechargeability, and *Bluetooth®* streaming in a one-of-a-kind design.

Available colors



IP68



Portable Charger (Default)
Holds 4 days of charging.
17 hours on a single charge.

Dry&Clean Charger

Qi Wireless Technology¹

Charger equipped with Qi Wireless Technology — enables wireless charging on any standard charging pad, including those built into cars and cafe counters.

Key features & benefits



My
WellBeing



iPhone &
Android
Connectivity



Recharge
on-the-go



CROS



Augmented
Speech
Understanding



Immersive
Soundscape



Automatic
Situation
Detection



24/7 Signia
Assistant



Auto
EchoShield



Own Voice
Processing
2.0



HandsFree with
CallControl
for iOS

- Receivers: miniReceivers 3.0 S,M,P

- Earwear: Earwear 3.0



red dot design award
winner 2019



¹ Wireless charging pad not included with purchase

² Requires StreamLine Mic

CROS — RICs



**CROS Pure
Charge&Go
T IX/AX**



**CROS Pure
312 AX**



**CROS
Styletto AX**



**CROS Pure
Charge&Go X**
compatible with
Motion C&G X

- Li-ion rechargeability and auto on/auto off function when removed from charger for easy handling.
- 29 hours of battery life for CROS Charge&Go and 90 hours of battery life for CROS 312.
- World's first CROS/BiCROS technology to benefit from acoustic-motion sensors for the clearest speech understanding that adapts to the wearer's movements (with BiCROS).
- Own Voice Processing (OVP™) (with BiCROS).
- Directional pattern skewed toward the unaidable ear for improved speech understanding.
- CROS mixing-level adjustment available in the Signia App to balance spatial benefit vs. noise level.
- Compatible with Signia App, StreamLine™ TV, StreamLine™ Mic, and miniPocket™.

**DID YOU
KNOW?**

Signia leads the industry in CROS/BiCROS technology.

A survey* among hearing care professionals found that:

- Signia CROS technology received an average of **4.5 out of 5 stars**, the majority stating it is the **BEST TECHNOLOGY** on the market.
- Most reported that they and their patients were **EXTREMELY SATISFIED** with Signia CROS.


When asked for specific feedback, here's what your peers had to say:

- "It is **the only CROS product that I have had success with**. These patients still have some challenges, but overall they **accept the fitting at a much higher rate than any other CROS product I have tried.**"
- "Patient is **hearing in situations they never heard** with their old [competitive CROS product]."
- Although we have used [competitor] in the past, my **previous [competitor] users who now have the Signia CROS are happier.**






Signia RIC products and fitting ranges

miniReceiver 3.0

6 Lengths

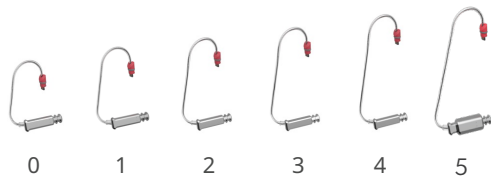

4 Power Levels

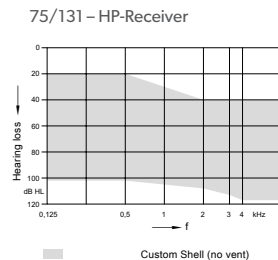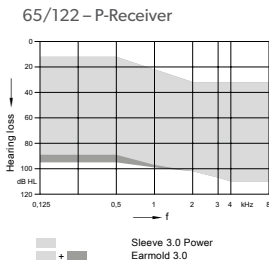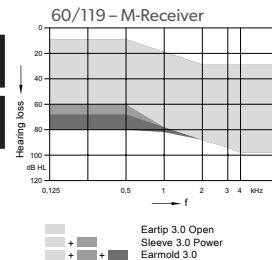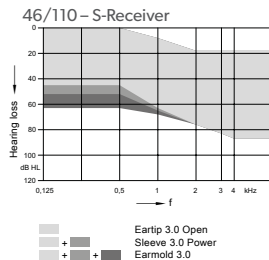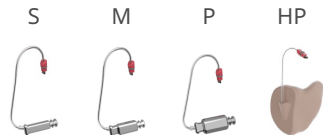

miniReceiver 3.0

receiver types

Our new EarWear 3.0 offers a higher physical fit rate, improved cosmetics and better retention



miniReceiver 3.0 Cable Bending
10 / 10 fit correctly – out of the box

Smaller plug for smaller RICs

Push-/pull system –
no tool required

Smart Receiver Detection
prevents misuse


6-Pin Connector for
new features

New bending ensures
higher retention in ear

New bending sits closer
= cosmetically more
appealing

Size 0-4

Reinforced cable outlet for
highest durability


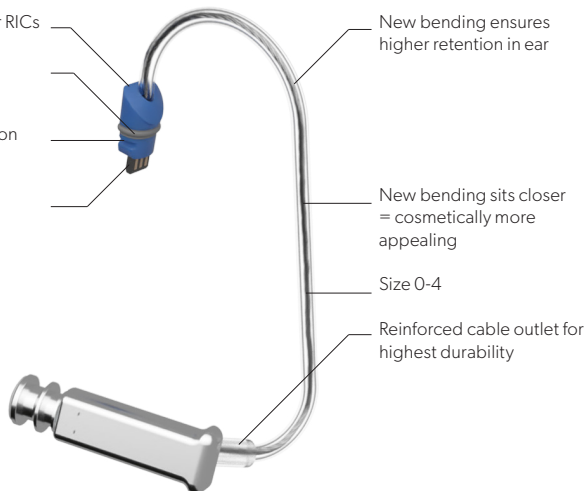


EarWear 3.0

This versatile new range is suitable for the vast majority of ear anatomies and hearing needs

Soft eartips allow for deep insertion with high comfort - also into small ear canals

- The new Power Sleeve 3.0 expands on the the acoustically reliable sleeve design, now offering a fully closed option.
- Secure, proven design makes Eartips 3.0 and Sleeves 3.0 much easier for wearers to exchange.
- All Earwear 3.0 items can be used together.

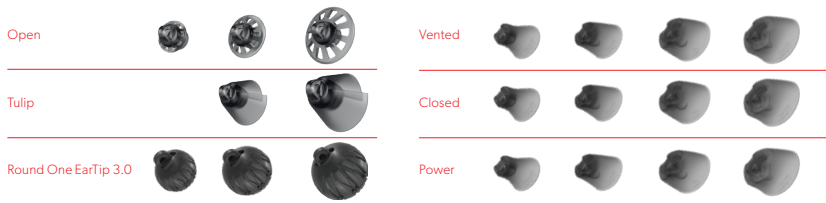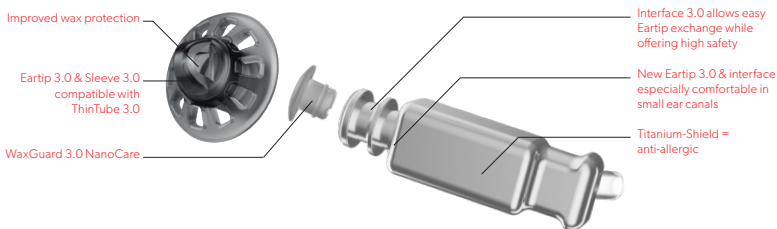

Motion Charge&Go X

Complete BTE lineup with the world's first rechargeable Super Power

New EarWear 3.0 thin tubes, domes and sleeves are required for thin tube fittings



Standard



Power



Super Power



IP68



Signia Motion
Charge&Go X



Signia Dry&Clean
Charger for P & SP

Up to 61 hours of use on a
single charge with SP, 30 hours
for P and 24 hours for Standard
(based on a full 4-hour charge)



Beige



Granite



Grey



Black



Silver
Hero



Pearl
White



Sandy
Brown



Dark
Champagne



Deep
Brown



Rose
Gold

Key features & benefits



Lithium-Ion



iPhone &
Android
Connectivity



Integrated
Telecoil



CROS



Tinnitus
Relief



Dynamic
Soundscape
Processing



Acoustic
Motion Sensors



Own Voice
Processing



Signia
App




Face Mask
Mode




AI Digital
Assistant

Motion Charge&Go X fitting ranges


Motion C&G X Fitting Range

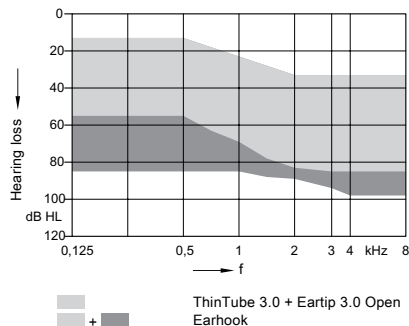

Motion C&G P X Fitting Range

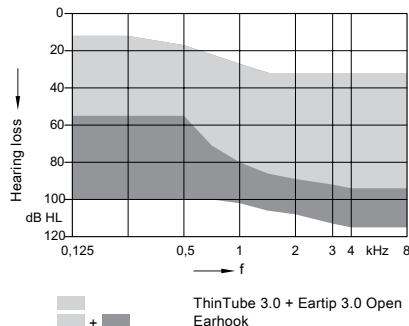

Motion C&G SP X Fitting Range


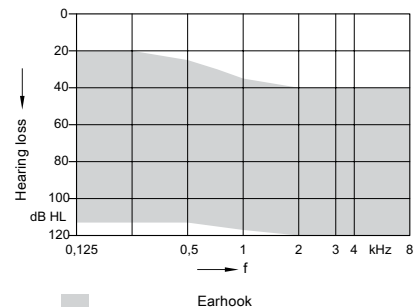


Thin Tube 3.0

Compatible with Motion Charge&Go X and P BTE

Both ThinTube 3.0 options offer proven better retention, comfort, usability and physical fit rate

The new 1.4mm-width PowerTube also delivers a cosmetically appealing solution for more severe degrees of hearing loss.

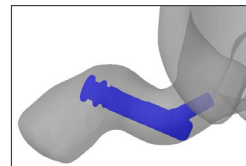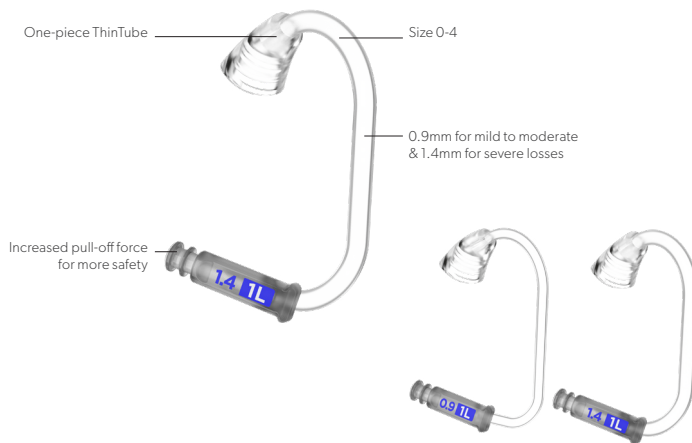








Retention

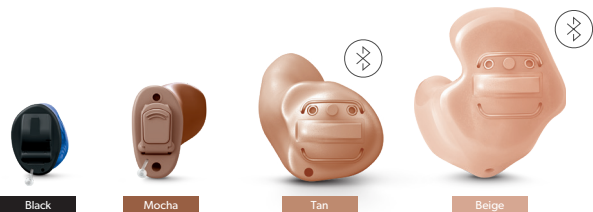Improved bending of tube + outlet locks the acoustical coupling reliably into the ear canal, prevents lateral migration and makes concha-locks redundant.

Binaural Thin tube 3.0 S0	10991609
Binaural Thin tube 3.0 S1	10991610
Binaural Thin tube 3.0 S2	10991611
Binaural Thin tube 3.0 S3	10991612
Binaural Thin tube 3.0 S4	10991613
Binaural Thin tube 3.0 S0 Power	10992492
Binaural Thin tube 3.0 S1 Power	10992493
Binaural Thin tube 3.0 S2 Power	10992494
Binaural Thin tube 3.0 S3 Power	10992495
Binaural Thin tube 3.0 S4 Power	10992496

Insio Nx Custom




Custom Faceplate Colors		Non-Bluetooth® Faceplate available			
Description	ITE	ITC/HS	CIC	IIC	
 Beige	●	●			
 Tan	●	●			
 Mocha	●	●		●	
 Brown	●	●		●	
 Dark Brown	●	●			
 Black				●	

Custom Shell Colors		ITE	ITC/HS	CIC	IIC
Description					
Beige	•	•			
Tan	•		•		
Brown	•	•	•		
Dark Brown	•	•			
Clear	•	•	•		
Red	•	•	•	•	
Blue	•	•	•	•	•

Programmers



 Noahlink Wireless
for ITC/HS and ITE

 ConnexxAir
for IIC and CIC



Stream audio directly into
your *Bluetooth®*-enabled Insio
ITC/HS and ITE instruments
with **Streamline™ TV** and
Streamline™ Mic

Insio Nx Custom

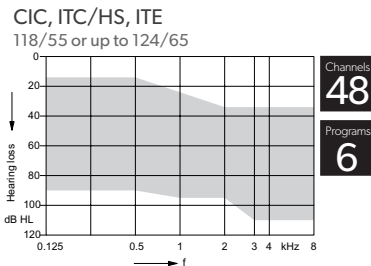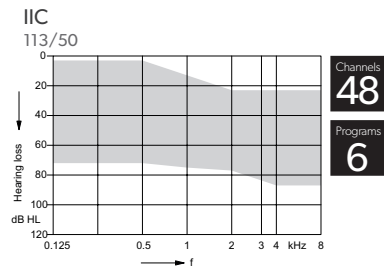

Motion 13 and 13 P Nx



Motion 13P Nx



Battery Type 13

Motion 13 Nx




Battery Type 13

IP68

Fitting Ranges


70/139 – Motion 13 Nx

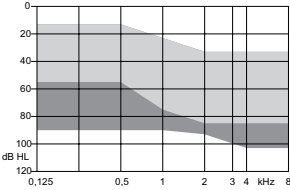

Channels
48

Programs
6

81/141 – Motion 13P Nx

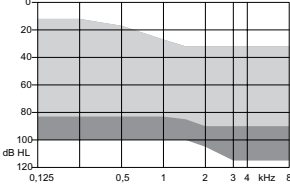

Channels
48

Programs
6



Beige



Granite



Grey



Black



Silver



Pearl White



Golden Blonde



Sandy Brown



Dark Champagne



Dark Granite



Deep Brown

StreamLine Mic – Compatible with all Signia Bluetooth® hearing aids

The new StreamLine™ Mic can be connected to virtually any Bluetooth-enabled smartphone (such as iPhone and Android) and establishes various direct audio streaming options:

- Phone call streaming (both ears)
- Integrated Microphone
- Hands-free phone calls (both ears)
- Accept/decline calls
- Remote microphone functionality (both ears)
- Hear in challenging situations with remote microphone mode
- Audio, video and music streaming (both ears, stereo)
- Remote Control functions
- Volume Increase
- Volume Decrease
- Program Change

"Made for iPhone" and "Made for iPad" mean that an electronic accessory has been designed to connect specifically to iPhone or iPad, respectively, and has been certified by the developer to meet Apple performance standards. Apple is not responsible for the operation of this device or its compliance with safety and regulatory standards. Please note that the use of this accessory with iPhone or iPad may affect wireless performance. iPad, iPhone are trademarks of Apple Inc., registered in the U.S. and other countries.

The Android name, the logo, and other trademarks are property of Google LLC.



Watch handling videos online
veterans.signiausa.com/howto

HOW TO ORDER IN ROES

ROES Main Menu – Patient Orders.

Select an Action: Wireless Devices

Remotes

Wireless Transmitter

Select Device Type: SIVANTOS

Select Make: StreamLine Mic

StreamLine Mic – Overview

Multi-function button (MFB)

- Switch on (press 3 sec) and off (press 6 sec)
- Toggle between two modes:
 - Bluetooth® Phone Mode
 - Remote Microphone Mode


Wearer controls/status indicators

- VC / Program up
- VC / Program down
- Power indicator
- Status indicator






StreamLine Mic offers intuitive onboard control and status elements

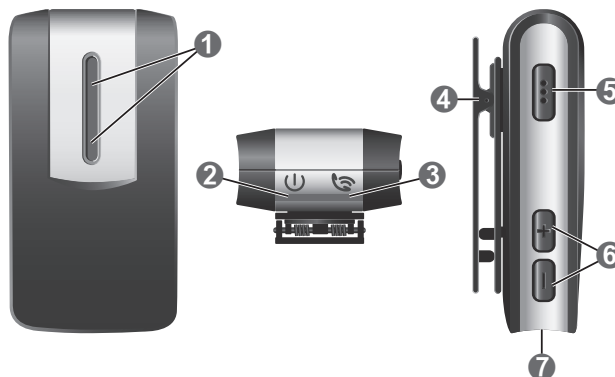Accessories included in the shipment

- 2 lanyards (regular and long)
- USB wall plug + USB connector
- Pouch
- User Guide



StreamLine Mic controls

- 1 Microphones
- 2 Power indicator 
- 3 Status indicator 
- 4 Lapel clip
- 5 Multi-function button (volume, program, mute) 
- 6 Volume buttons  and 
- 7 USB port for charging

StreamLine Mic – Pairing Quick Start Guide

Pairing with Hearing Aids

- Open/close the battery doors of the hearing aids and/or place in/out of the charger to place them in pairing mode.
- Turn on StreamLine Mic by holding down the MFB until the power indicator is green. This takes approximately 3 seconds. The status indicator flashes yellow.
- Place both hearing aids close to StreamLine Mic (maximum distance 20 cm or 8 inches). The pairing process starts automatically. Pairing is finished when the status indicator is solid yellow.
- A solid green power indicator and solid yellow status indicator verify connection to hearing aids is successful with no mobile device paired.
- Your hearing aids need to be paired once with StreamLine Mic.

Pairing with Smartphone

- Confirm that Bluetooth® of the mobile device is on.
- Turn on StreamLine Mic by holding down the MFB until the power indicator is green. This takes approximately 3 seconds.
- Hold down the the MFB and “+” button on the StreamLine Mic at the same time until status indicator flashes blue. This takes approximately 6 seconds.
- The wearer has 3 minutes to complete the pairing.
- Select "StreamLine Mic" from the list of detected devices on the Bluetooth® menu.
- If requested, enter the PIN "0000".
- A solid green power indicator and solid blue status indicator verify connection to smartphone is successful.



StreamLine Mic – Battery Runtime

- Charging status is clearly displayed via power indicator on the StreamLine Mic
- Full charge takes approx. 2 hours
- Up to 6 hours battery runtime after full charge with full functionality (phone calls, audio streaming, remote mic)
- Can be charged via USB or usual plug-in (all necessary cables are included in the delivery)
- Can be used while charging

StreamLine TV – Compatible with all Signia Bluetooth® hearing aids

The perfect companion
for TV fans.

StreamLine TV.

Thanks to StreamLine™ TV audio from television or any device with audio output can be fed directly into Signia hearing aids. Wearers can even adjust the volume individually via the Signia app.

All wearers of IX, AX, X and Nx hearing aids have direct binaural streaming for TV via the StreamLine TV.

This includes use without the Signia app for non-smartphone users.



HOW TO ORDER IN ROES

ROES Main Menu – Patient Orders.

Select an Action: Wireless Devices

Remotes

Wireless Transmitter

Select Device Type: SIVANTOS

Select Make: StreamLine TV

Pairing for StreamLine TV

If ordered together, Signia will pair all Bluetooth® compatible hearing aids to the Streamline TV.

The hearing aids need to be paired once with the transmitter. After that, the pairing is saved in the hearing aids and remembered each time they are turned on.


- Ensure that the transmitter is connected to the power supply.
- Turn the hearing aids off and on again: open and close the battery compartments. If your hearing aids do not have a battery compartment, turn them off and on as described in the hearing aids' user guide.
- Place both hearing aids close to, or on top of, the transmitter. The pairing process starts automatically.
- Observe the indicators:



Description



If pairing is completed, both indicators are blue for about 30 seconds.

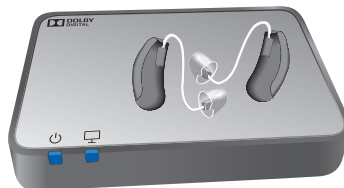


Multiple users:

- The transmitter can be paired to multiple hearing aids. All hearing aids will be able to hear the same audio signal.
- To clear the pairings of one current user, first reset the transmitter to factory settings. This will clear the pairings of all users. After that, pair the hearing aids of the other users again. Refer to the section "Resetting to factory settings".

Connecting StreamLine TV

TOSLINK connection

- Before connecting the cable, remove the protection caps from the TOSLINK cable plugs.
- Plug the cable into the TOSLINK **input** of the transmitter and into the TOSLINK (optical audio) **output** of electronic device. To prevent the TOSLINK cable from breaking, do not forcefully bend it.

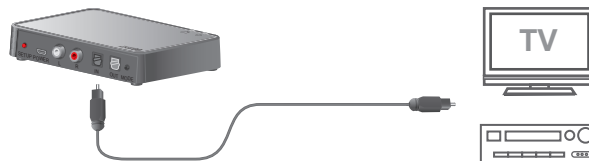


Further information

- Refer to the electronic device's manual: set the output format of the electronic device to either PCM (pulse-code modulation) or Dolby® Digital.*

RCA connection

- Plug the cable into the RCA connections of the transmitter.
- Plug the other side of the cable into the RCA output of the electronic device or via adapter into the SCART or via mini jack output of the electronic device.

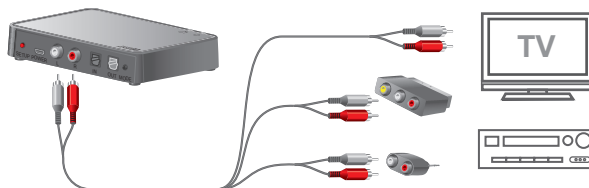

For more information and details on how to connect to your home theater amplifier or a sound bar watch the handling videos online at veterans.signiausa.com/howto

*Dolby and the double-D symbol are registered trademarks of Dolby Laboratories.


miniPocket — Remote control compatible with all Signia hearing aids

Small enough to hang on a keyring, the miniPocket™ is ideal for wearers who want quick, discreet control of their hearing aids without using a smartphone.

- Battery type CR2450
- Pairing remains in place when battery dies and is replaced

Components

- ❶ Status LED
 - ❷ Volume buttons
 - ❸ Keyring
 - ❹ Key lock switch
 - ❺ Program change button
-

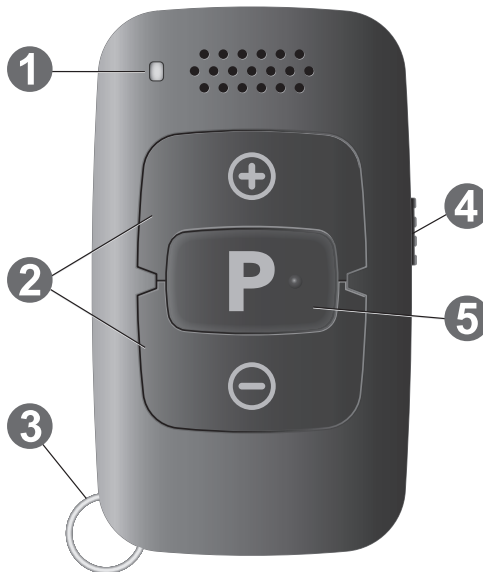

Pairing hearing aids with miniPocket

1. Move key lock switch to 'lock' position



2. Turn on hearing aids and place in ears
3. Hold down **P** button on the remote control while switching the key lock to the unlock position
4. Release **P** button (LED on)
5. Press and hold **P** button until the pairing confirmation tone is heard
6. Move Key Lock switch to Lock position to exit pairing mode



Watch video online

veterans.signiausa.com/howto

HOW TO ORDER IN ROES

When ordering a hearing aid in ROES, look for the "Select a remote" dropdown. Select "miniPocket"

Rechargeability



Pure Charge&Go T IX/AX Standard Charger

- Compatible with Pure Charge&Go T IX, Pure Charge&Go T AX and CROS Charge&Go AX
- Convenient, table-top design



Pure Charge&Go T IX/AX Portable Charger

- Compatible with Pure Charge&Go T IX, Pure Charge&Go T AX and CROS Charge&Go AX
- Portable, closed-lid charger
- Holds three charges



Insio Charge&Go AX Charger

- Compatible with Insio Charge&Go AX featuring contactless charging



Closed-lid Inductive Charger

- Compatible with Pure Charge&Go X and Motion Charge&Go X
- Electronically dehumidifies the hearing aids
- Room for earmolds



Styletto AX Charger

- Compatible with Styletto AX
- Portable, closed-lid charger equipped with Qi Wireless Technology—enables wireless charging on any standard charging pad, including those built into cars and cafe counters*



Dry&Clean Charger

- Dry&Clean Chargers are available for Motion C&G P, Motion C&G SP, and Pure C&G T AX. This charger will simultaneously charge, electronically dehumidify, and sanitize the hearing aids and custom molds with the integrated UV-C light.
- Compatible with Motion X, Pure IX, Pure AX, and Styletto AX

*Wireless charging pad not included with purchase

Portable IX/AX Charger

For Pure Charge&Go T IX, Pure Charge&Go T AX,
and CROS Charge&Go AX



NOTICE

Please make sure that the power plug
is easily accessible to remove it from
power supply if necessary.

- Connect the charger to the power supply. All LEDs for each charging slot turn on successively indicate the charger has power and is ready for use.
- Put the hearing aids in the charging slots as shown in the picture. You can charge both hearing aids at the same time. The hearing aids are automatically turned off and start charging. The charging status is indicated, see table "Charging status".
- When you want to use your hearing aids, take them out of the charger. They will automatically turn on.



Charger Lights

LED when charger is charging:

Pulsing green: internal battery is being charged.

Solid green: charger is fully charged.

Blinking red: charger error. Please refer to section "Troubleshooting".

LED when you are mobile:

Solid green: the internal battery has enough capacity to fully charge a pair of hearing aids.

Solid orange: the internal battery does not have enough capacity to fully charge a pair of hearing aids.

Blinking orange: the internal charger battery is empty. Charging of hearing aids is not possible.

Insio Charge&Go Charger

For Insio Charge&Go AX



NOTICE

Please make sure that the power plug is easily accessible to remove it from power supply if necessary.

- Connect the charger to the power supply. All LEDs for each charging slot turn on successively indicate the charger has power and is ready for use.
- Put the hearing aids in the charging slots as shown in the picture. You can charge both hearing aids at the same time. The hearing aids are automatically turned off and start charging. The charging status is indicated, see table "Charging status".
- When you want to use your hearing aids, take them out of the charger. They will automatically turn on.



Charging status

LED	Status description
	No LED: Charger is not connected to power supply.
	Single green LED turns on successively: Charger is connected to a power supply and ready to charge. No hearing aid is inserted in the respective charging slot or the hearing aid is not inserted correctly.
	Flashing orange LED: Charging status of the hearing aid is read out before charging starts.
	Single orange LED: Charging status 0 to 33 %
	Turn on single green LED: Charging status 34 to 66 %
	Turn on two green LEDs: Charging status 67 to 99 %
	Three green LEDs: Hearing aid is fully charged.
	Single orange LED flashes successively: Cooling phase Charging is resumed after 5 minutes.
	Flashing red LEDs: Error Read more in section "Troubleshooting".

Closed-lid Inductive Charger

For Pure Charge&Go X, CROS Charge&Go X, Motion Charge&Go X BTE



NOTICE

Please make sure that the power plug is easily accessible to remove it from power supply if necessary.

- Connect the charger to the power supply. All LEDs for each charging slot turn on successively indicate the charger has power and is ready for use.
- Put the hearing aids in the charging slots as shown in the picture. You can charge both hearing aids at the same time. The hearing aids are automatically turned off and start charging. The charging status is indicated, see table "Charging status".
- When you want to use your hearing aids, take them out of the charger. They will automatically turn on.




Charging status

LED	Status description
	No LED: Charger is not connected to power supply.
	Single green LED turns on successively: Charger is connected to a power supply and ready to charge. No hearing aid is inserted in the respective charging slot or the hearing aid is not inserted correctly.
	Flashing orange LED: Charging status of the hearing aid is read out before charging starts.
	Single orange LED: Charging status 0 to 33 %
	Turn on single green LED: Charging status 34 to 66 %
	Turn on two green LEDs: Charging status 67 to 99 %
	Three green LEDs: Hearing aid is fully charged.
	Single orange LED flashes successively: Cooling phase Charging is resumed after 5 minutes.
	Flashing red LEDs: Error Read more in section "Troubleshooting".

Stylecto AX Charger

- Place the right hearing aid in the charging slot marked with an "R", and the left hearing aid in the charging slot marked with an "L".
- Stow the ear pieces nicely.
- Close the charger's lid for battery power saving reasons.
- The hearing aids are automatically turned off and start charging. The charging status is shown for 7 seconds.

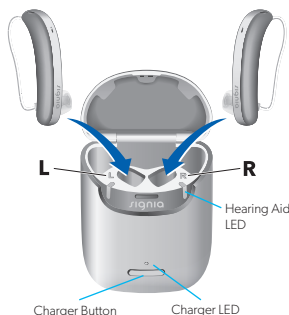

If the status LEDs do not light up when you insert the hearing aids, check the following:

- The hearing aids are not correctly placed. Push them slightly into their charging slots or close the charger's lid.
- The charger is turned off. Press and hold the button for 5 seconds to turn on the charger.
- The charger's internal battery is too low for mobile charging. Connect the charger to an electrical outlet.



NOTICE: If you put a hearing aid in the wrong charging slot, the receiver cable may be damaged when closing the charger's lid.

Charging status of hearing aids

The LED next to the hearing aid indicates the charging status of the hearing aid. Note that the status is only displayed immediately after putting the hearing aids in the charger, when you briefly press the button or when you open the lid. When the charger is off, no status is displayed at all.

Hearing Aid LED Status description

Pulsing green: Hearing aid is being charged.

Green: Hearing aid is fully charged. Internal battery has enough capacity to fully charge two hearing aids.

Charger LED Status description

LED is off: No hearing aid is in the charger, hearing aid is not placed correctly, or charger is off.

Pulsing orange: Cooling phase. Charging is resumed when the device has cooled down. Please refer to section "Troubleshooting".

Blinking red: Hearing aid error. Please refer to section "Troubleshooting".

Green: Internal battery has enough capacity to fully charge two hearing aids.

Dry&Clean Charger

For Pure Charge&Go T IX, Pure Charge&Go AX, Styletto AX, Motion P X and Motion SP X



Cleaning and Drying

- Drying and cleaning focuses on tubes, ear tips and molds.
- Put the hearing aids into the charging slots and close the lid.
- UVC cleaning starts automatically after closing the lid and lasts 15 minutes.
- Ongoing UVC cleaning process is indicated by LEDs (see table "Charging and cleaning status").
- Drying starts automatically after closing the lid and lasts 4 hours.


Charging

A full charging cycle takes approximately 3 to 4 hours.





After 30 minutes, it is possible to use your hearing aids for up to 7 hours.

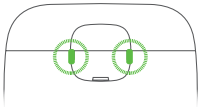


Charging times may vary due to the use of the hearing aids and the age of the power cell.


Dry&Clean Charger






CHARGE INDICATORS

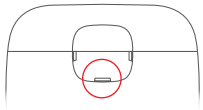

Solid yellow 	The charger is detecting the hearing aid.
Flashing green 	The hearing aid is charging, low battery, less than 40%.
Glowing green 	The hearing aid is charging, battery more than 40%.
Solid green 	The hearing aid is fully charged, battery 100% charged.

One indicator flashing red 	An error occurred during charging, and the charging has stopped.
Both indicators flashing red 	An error occurred during charging, and the charging has stopped.
Flashing yellow 	The temperature of the hearing aid has become too high during charging, and charging is paused.



DRY AND CLEAN INDICATORS

Solid yellow 	The maintenance cycle is paused, but the charging cycle continues.
Flashing white 	The maintenance cycle is in progress (drying/UV-C).
Solid white 	The maintenance cycle is complete.

Flashing red 	An error occurred during the maintenance, the cycle has stopped.
Flashing yellow 	The maintenance cycle is paused due to overheating (fan still running).

How to order a Charger in ROES

Choose from Other Commodities, and follow these ordering steps:

ROES Main Menu – Patient Orders

Select an Action: Other Commodities or Accessories

In Commodity Group pull-down menu, choose: SIVANTOS

In Vendor pull-down menu, choose:

In Search box please enter one of the following words:

ROES Name	Description	Part Number
Insio Charge&Go AX Charger	Charger for Insio Charge&Go 7 AX	10997503
Pure Charge&Go IX/AX Portable Charger	Compatible with Pure Charge&Go T AX, CROS Pure AX	10993118
Pure Standard Charger	Charger for Pure Charge&Go T AX, CROS Charge&Go AX	
Pure Dry&Clean Charger	Charger for Pure Charge&Go T AX, CROS Charge&Go AX	
Charger Styletto AX	Charger for Styletto AX, CROS Styletto AX	10972015
Inductive Charger II	Charger for Pure Charge&Go X, CROS Pure X, and Motion Charge&Go X	10966942
Dry & Clean Charger Motion X BTE P	Dry & Clean Charger for Motion Charge&Go P X BTE	10991638
Dry & Clean Charger Motion X BTE SP	Dry & Clean Charger For Motion Charge&Go SP X BTE	10991644
Smart Lithium-ion Dryer	NX and older charging options	10960141

NOTE If you want it delivered to your station, then change the “Deliver To” shipping address, otherwise, it will ship to the patient by default. Please note: a charging unit comes shipped with every Li-ion hearing aid order.

Portable Power Supply




- The Portable 2600mAh Mobile Power Bank is the perfect power solution when on the road or stuck somewhere without convenient electricity.
- It fits comfortably in a pocket, purse, backpack, or laptop bag to keep Pure Charge&Go or Motion Charge&Go hearing aids charged when needed.
- Capacity of 2600mAh will provide three full charges.



Portable Power Supply (10948100)

RIC Earmolds

Bold text indicates default options

	RIC Earmold 3.0		RIC Embedded Earmold 3.0	RIC HP Earmold 3.0
				
Styles	RIC Hollow (Hard Material Only) RIC Lock Hollow (Hard Material Only) Half Shell Hollow (Hard Material Only) Skeleton Hollow (Hard Material Only) RIC RIC Lock		RIC Solid RIC Lock Solid	Canal Canal Lock Helix Lock
Receivers	Receiver 3.0 S (Size 0-5) Receiver 3.0 M (Size 0-5) Receiver 3.0 P (Size 0-5)		Receiver 3.0 S (Size 0-5) Receiver 3.0 M (Size 0-5) Receiver 3.0 P (Size 0-5)	Receiver 3.0 HP (Size 0-5)
Materials	Hard Soft		Hard	Hard
Colors	Gloss and Matte Finishes Available Clear , Rose, Brown (Hard Material Only), Medium Brown, Dark Brown		Gloss and Matte Finishes Available Clear , Rose, Brown (Hard Material Only), Medium Brown, Dark Brown	Beige, Mocha, Brown, Red/Blue, Clear *Red/Blue and Clear shells come with Mocha faceplate
Venting	None Open (Hollow Only) Pressure (1.0MM) 1.2MM 1.6MM		None Pressure (1.0 MM) 1.6MM	None Pressure (1.0 MM) 1.6 MM
Ear Canal Length	Short, Medium , Long (Hard Material Only)		Short, Medium, Long	Short, Medium, Long, Deep
Indicators	Red/Blue Dots Patient Name Red/Blue Letters		Red/Blue Dots Patient Name Red/Blue Letters	N/A
Dexterity	Removal Line		Removal Line	Removal Line
Wax Protection	NanoCare 3.0		NanoCare 3.0	Cerustop 3.0 XL

BTE Earmolds

Bold text indicates default options

	BTE THIN TUBE 3.0 Earmold		BTE Conventional Earmold	
				
Styles	Canal Hollow (Hard Material Only) Canal Lock Hollow (Hard Material Only) Canal Canal Solid Helix Lock Solid	Full Shell Solid Half Shell Solid Semi-Skeleton Solid Skeleton Solid	Canal Solid Canal Lock Solid Helix Lock Solid Full Shell Solid	Half Shell Solid Semi-Skeleton Solid Skeleton Solid
Tubing	Thin Tube 3.0 (Size 0-4) Thin Tube 3.0 Power (Size 0-4)		Standard Thick Thick Dry Extra Thick Libby Horn	Standard Double Bend Clear Thick Double Bend Clear Thick Dry Double Bend Clear Extra Thick Double Bend Clear
Materials	Hard	Soft	Hard	Soft
Colors	<u>Gloss and Matte Finish Options</u> Clear, Rose, Beige, Brown, Red, Blue	Clear Gloss is default	<u>Gloss and Matte Finish Options</u> Clear, Rose, Beige, Brown, Red, Blue	Clear Gloss is default
Venting	None Open (Hollow Only) Pressure (1.0 MM) 1.2 MM 1.6 MM	SAV (2.0 MM) SAV (3.0 MM) Large as Possible Trench Vent (Hard Material Only) Semi-IROS	None Pressure (1.0 MM) 1.2 MM 1.6 MM SAV (2.0 MM)	SAV (3.0 MM) Large as Possible Trench (Hard Material Only) Semi-IROS
Ear Length	Short, Medium , Long (Hard Material Only)		Short, Medium , Long	
Indicators	Red/Blue Dots Red/Blue Letters	Patient Name	Red/Blue Dots Red/Blue Letters	Patient Name
Dexterity	Removal Line		Removal Line	
Wax Protection	NanoCare 3.0		None	


Signia Nomenclature

The names of our hearing aids are clues to what's under the hood....







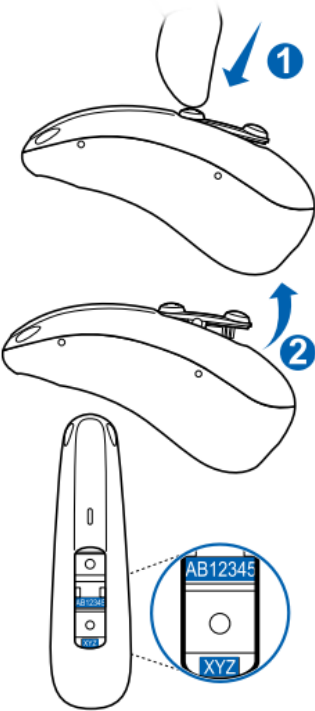

Make	Style	Model	Feature
Pure	RIC	C&G IX T	Rechargeable Integrated Xperience Telecoil
Styletto	Slim RIC	C&G AX	Rechargeable Augmented Xperience
Insio	Custom	C&G AX	Rechargeable Augmented Xperience
Motion	BTE, P BTE, SP BTE	C&G X	Rechargeable Xperience
Pure	RIC	312 AX	Battery AX Platform
Insio	Custom	NX	Battery NX platform
CROS	RIC	C&G, 312, Styletto	IX transmits to IX AX transmits to AX X transmits to X C&G transmits to C&G 312 transmits to 312

Where to Find Serial Numbers

C&G RIC	Under the Rocker Switch
RIC 312	Under the Rocker Switch
Customs	Printed on shell
Styletto AX	Underside of aid
Streamline™ Mic	Under swivel clip
Streamline™ TV	Under SLTV
MiniPocket™	Sticker in battery compartment



Charger Compatibility










Chargers		Integrated Xperience (IX) and Augmented Xperience (AX)							
		Pure Charge&Go Portable Charger*	Pure Charge&Go Charger	Pure Charge&Go Dry&Clean Charger	Insio Charge&Go AX Charger	Styletto AX Charger*	Styletto AX Dry&Clean Charger	Charging Pad ¹	Clinic Multicharger
Type of charger		Galvanic	Galvanic	Galvanic	MR Inductive	Galvanic	Galvanic	–	Galvanic
Part number		10993118	10996345	10991650	10997503	21003216	21007593	PN 10245297	PN 21026076
									
Cord connection		USB-C PN 21023081	Attached	USB-C PN 21023081	USB-C PN 21023081	USB-C PN 21023081	USB-C PN 21023081		Attached
Wear time without streaming		39h	39h	39h	24h	20h	20h		39h
Wear time with streaming		36h	36h	36h	20h	17h	17h		36h
Charge time		4h	4h	4h	4h	4h	4h		4h
Quick charge time		30m for 9h	30m for 9h	30m for 9h	30m for 4h	30m for 5.5h	30m for 5.5h		30m for 9h
Portable charger extra charges		3	N/A	N/A	N/A	4	N/A		N/A
Hearing Aids									
RIC	Pure C&G IX/AX	*	•	•					•
	CROS Pure C&G IX/AX	*	•	•					•
	Styletto AX					*	•	•	•
	CROS Styletto AX					*	•	•	•
Custom	Insio C&G AX				••				

* Compatible • Default (Recommended)

Please note – While the X hearing aids will work with the Inductive Charger I, the charger was not custom made for their shape.

Please verify that the lights turn on when placing the hearing aid in the charger so that you can ensure proper placement.

Charger Compatibility

Chargers	Xperience (X)								
	Inductive Charger I	Inductive Charger II	Styletto X Charger	Dry&Clean Charger P X	Motion C&G Charger P X	Dry&Clean Charger SP X	Motion C&G Charger SP X	Charging Pad ¹	Clinic Multicharger
Type of charger	Inductive	Inductive	Galvanic	Galvanic	Galvanic	Galvanic	Galvanic	–	Galvanic
Part number	10948199	10966942	10972015	10991638	10996335	10991644	10996340	PN 10245297	PN 210265076
									
Cord connection	micro USB	micro USB	USB-C	USB-C	Attached	USB-C	Attached		Attached
Wear time without streaming		24h	19h	30h	30h	61h	61h		39h
Wear time with streaming		21h	16h	27h	27h	57h	57h		36h
Charge time		4h	4h	4h	4h	4h	4h		4h
Quick charge time		30m for 6h	30m for 5h	30m for 7h	30m for 7h	30m for 4h	30m for 4h		30m for 9h
Portable charger extra charges		N/A	3	N/A	N/A	N/A	N/A		N/A
Hearing Aids									
RIC	Pure C&G X	•	••						
	CROS Pure C&G X	•	••						
	Styletto X			••				•	
	Pure C&G Nx	•	••						
BTE	Motion C&G X	•	*						
	Motion C&G P X			••	•				•
	Motion C&G SP X					••	•		•


• Compatible • Default (Recommended)

Please note – While the X hearing aids will work with the Inductive Charger I, the charger was not custom made for their shape.





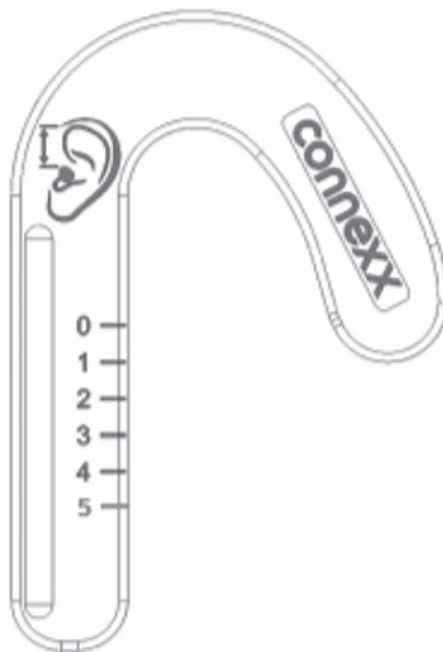

Please verify that the lights turn on when placing the hearing aid in the charger so that you can ensure proper placement.

How to Measure for Signia 3.0 Receivers

1. Use Measuring Gauge for EarWear 3.0 (white)
2. Place the measuring gauge on the ear of the wearer.
3. Read the number that comes closest to the upper portion of the ear canal opening.



Signia Earwear 3.0 Parts

Item		Description	Brand
Measurement Gauge 3.0		Ensure perfect match for new miniReceiver/Thin Tube 3.0 in 5 lengths (0-4)	Connexx
Wax Guard 3.0 NanoCare – miniReceiver 3.0		Matchbook design containing 8 wax guards	Connexx, Signia
CeruSTOP 3.0 XL – for HP Earmolds			
Concha Locks Sleeves 3.0		New one-size-for all Concha lock. 10 pieces per bag	Connexx
Removal Tool 3.0		Tool to remove miniReceiver/Thin Tube from Earmold 3.0 Short/Long	
Handling Tool 3.0 for custom eartip		Tool to press out the receiver from Earmold 3.0 Short	

Signia Wax Guard Guide



miniR Wax Guard
Part #10966257
Used for: 2.0 Receivers



3.0 NanoCare
Part #10993607
Used for: 3.0 Receivers, 3.0 Earmolds, Silk C&G IX CIC



QuickGuard
Part #10962504
Used for: Insio Customs and 2.0 Earmolds




3.0 CeruSTOP XL
Part #10999334
Used for: 3.0 HP Receivers




Earmold Ordering Tips

- Order 3.0 Earmolds for all IX and AX receivers and Motion X Thin Tubes
- 3.0 Hollow Earmolds are open earmolds (same as Foil Earmolds for the 2.0 receivers)
- All HP receivers need an impression as they have to have earmold
- Our CROS has a real receiver and any earmold will work. We suggest Hollow for 3.0 earmolds or Foil for the 2.0 earmolds

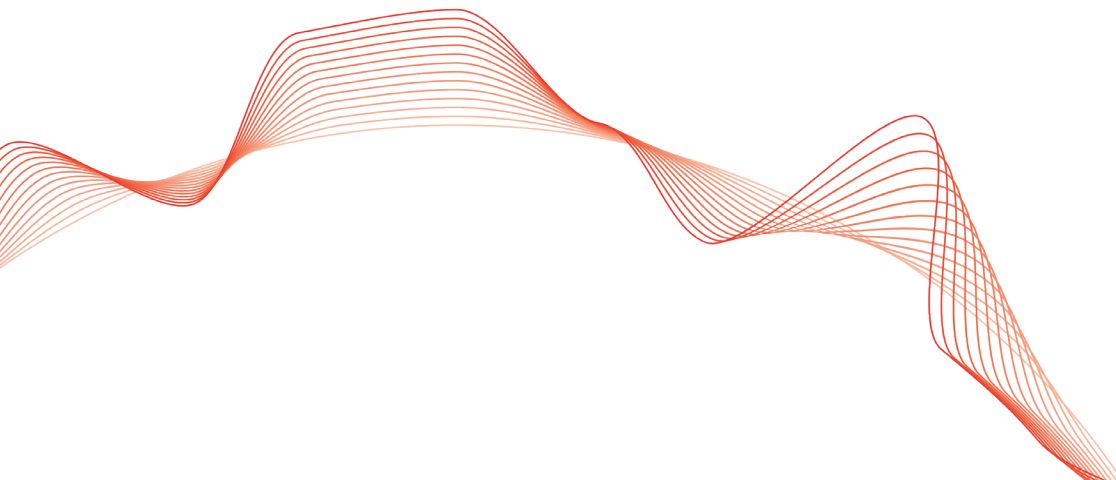

Accessories Tips

- The miniPocket takes a CR2450 battery – this is labeled as an “L2450” battery in ROES for re-ordering
- Current accessories do not have to be connected/paired to the aids in the software
- The medium press of the hearing aid push button/rocker switch is defaulted to TV Stream for easy access

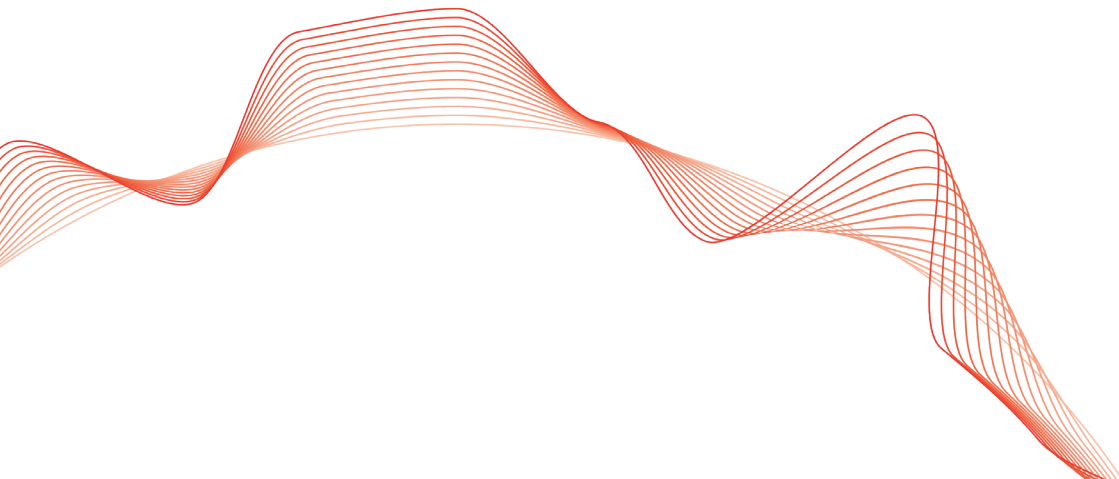

Signia Fitting Guide

NOTE: All ordered hearing aids have already been First Fit.

1. Select your patient from NOAH patient database
2. Launch Signia Connexx fitting software
3. Select "Detect Hearing aids"



Audiogram




Detect Hearing Instruments




Hearing Loss Simulator

4. Once hearing aids are detected, read out connected devices

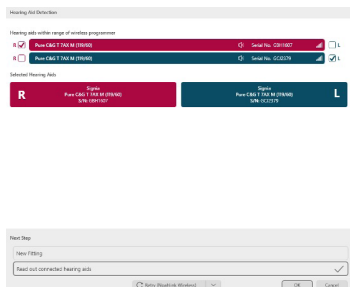


Please note:

- a. Since aids have been first fit, if you are reading out connected instruments (recommended) Connexx will open on the Data Log screen.
- b. Visually confirm green check mark beside First fit.

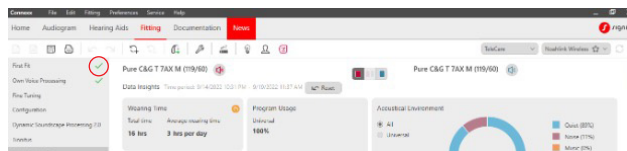

*Note: Bone conduction is considered for fitting. If Audiogram in Noah does not show bone conduction, then you may get an audiogram error message.

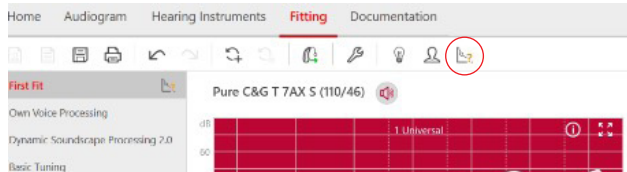


- c. If any acoustic parameters are adjusted on the First Fit tab, we recommend performing a new first fit.

Signia Fitting Guide


(continued)

5. Run OVP (Own Voice Processing)*

For optimal results

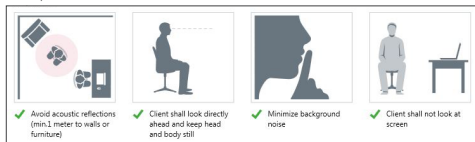

Complete the OVP Training. Tell your patient that they will hear a beep and that they should count loudly and continuous from 21 upwards. Example: 21, 22, 23, and so on.

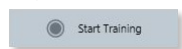

OVP is automatically enabled. Select "OK".  Training was successful

*OVP is not available with customs.

6. Recommended Best Practices for adjusting gain to Real Ear Targets

a. While aids are coupled adjust overall gain for aids in Fine Tuning > Sound & Loudness, Frequency Shaping tab under gain curve screen.



b. If further fine tuning is needed uncouple Right and Left said go into Gain tab under gain curve and adjust soft moderate and loud gain curves independently.



Signia Fitting Guide


(continued)

7. Select Configuration Configuration

- **Hearing aid** – Configure User Controls, Volume Control, Therapy Signal, SoundBalance.
- **System Sounds** – Configure Alert Tones.


8. Save Session and Select “X” at top right of screen or select drop down item under File.

- Disconnect hearing aids.

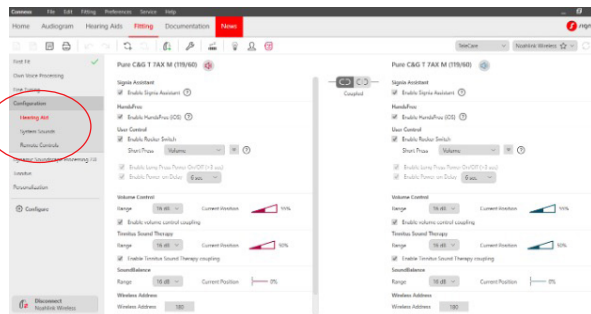

Tinnitus Fitting Guide

Notch Therapy is only available in Signia hearing instruments. Notch Therapy can be customized to treat the patient's tonal tinnitus in a few simple steps with Connexx® fitting software. Find the patient's pitch match by using Guided Matching, Manual Matching, or choosing a known pitch from Direct Entry. Perform the frequency check to verify the correct octave of the pitch match and finally apply the notch to desired programs. It's that quick and easy.

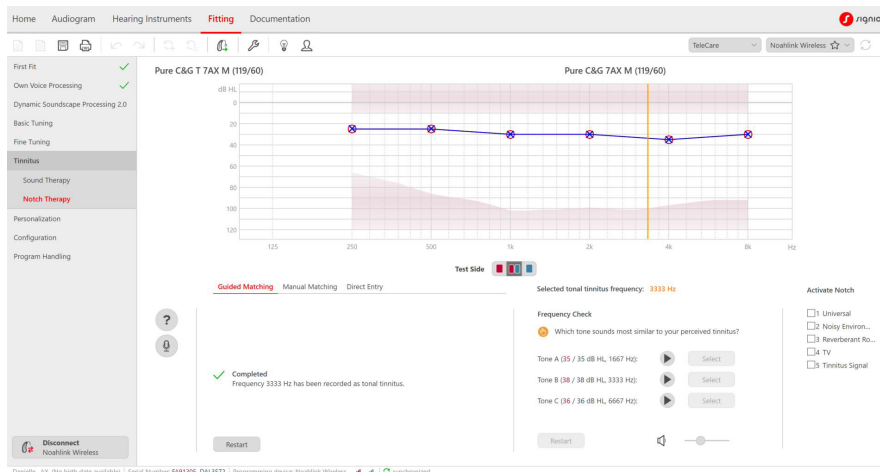

CROS Pure Charge&Go T IX, AX, X and CROS Pure 312 AX Quick Fit Guide

Under Fine Tuning, select Sound Settings to adjust the sensitivity of the CROS microphone in relation to the hearing aid mic.


If the “Enable CROS microphone” is unchecked, the CROS microphone will be disabled.

Input Configuration

Mode: CROS

☒ Enable CROS microphone

CROS level


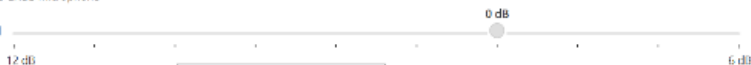


Adjust the CROS microphone level

CROS/BICROS Listening Check

Outside of Connexx

1. Turn on Both Hearing aid and CROS/BICROS
2. Hearing Aids need to be "Head Distance Apart"
3. Rub CROS/BICROS Microphones

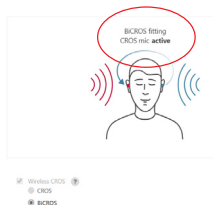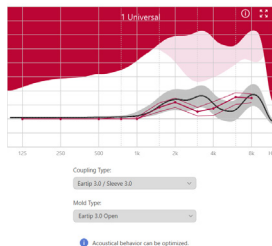

Inside Connexx Software

1. Detect/Connect to both Hearing Aid and CROS/BICROS
2. First Fit Settings will show CROS/BICROS Mic Active
3. Fine Tuning>Sound Settings > CROS Mic Slider Enabled

Input-Configuration

Mode: CROS



Enable CROS microphone

i CROS Microphone is not available in

CROS level

-12 dB

How to Pair

Pairing with an iPhone

1. Open the “Settings” app on the iPhone,
OR from the Signia app select “Bluetooth®”
connection option.
2. In the settings menu on your phone:
 - a. Select “Accessibility”
 - b. Select “MFi Hearing Devices” or
“Hearing Devices”
3. Ensure Bluetooth® is enabled and pair the
hearing aids when prompted; the names
of your hearing aids will be in blue when
successful.
4. Choose desired connection method.
5. Open the Signia app and tap “Let’s Go” to
pair your hearing aids to the app.

Pairing with an Android

1. When pairing your hearing aids to an Android
device, go directly into the Signia app (the
phone’s native Bluetooth® menu does not
allow correct pairing).
2. Open Signia app and select “Bluetooth®”
connection option.
3. Verify phone is on the list, tap “Next”.
4. Restart the hearing aids and place them next
to the phone. Tap “OK”.
5. Tap the names of your hearing aids once they
appear.
6. Tap “Let’s Go” to pair your hearing aids to
the app.



Visit signiausa.com/signia-app
for interactive user guide and
additional information:



ROES codes

To place an order for any of the following in ROES first choose **Commodities**, then **Parts and Accessories**, then search by item number.

VA ROES ITEM	DESCRIPTION	ITEM #	QTY.	PRICE	COMPATIBLE PRODUCTS
RECEIVERS					
MINI RECEIVER 3.0 KIT	MINI RECEIVER KIT 3.0 Complete	21018572	1	\$728.00	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX, PURE 312 AX, CROS PURE 312 AX, STYLETTO AX, CROS STYLETTO AX
MINIRECEIVER 3.0 45 dB SIZE 0 RIGHT EAR	MINIRECEIVER 3.0 45 dB SIZE 0 RIGHT EAR	10992548	1	\$51.36	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX, PURE 312 AX, CROS PURE 312 AX, STYLETTO AX, CROS STYLETTO AX
MINIRECEIVER 3.0 45 dB SIZE 0 LEFT EAR	MINIRECEIVER 3.0 45 dB SIZE 0 LEFT EAR	10992549	1	\$51.36	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX, PURE 312 AX, CROS PURE 312 AX, STYLETTO AX, CROS STYLETTO AX
MINIRECEIVER 3.0 45 dB SIZE 1 RIGHT EAR	MINIRECEIVER 3.0 45 dB SIZE 1 RIGHT EAR	10992550	1	\$51.36	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX, PURE 312 AX, CROS PURE 312 AX, STYLETTO AX, CROS STYLETTO AX
MINIRECEIVER 3.0 45 dB SIZE 1 LEFT EAR	MINIRECEIVER 3.0 45 dB SIZE 1 LEFT EAR	10992552	1	\$51.36	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX, PURE 312 AX, CROS PURE 312 AX, STYLETTO AX, CROS STYLETTO AX
MINIRECEIVER 3.0 45 dB SIZE 2 RIGHT EAR	MINIRECEIVER 3.0 45 dB SIZE 2 RIGHT EAR	10992553	1	\$51.36	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX, PURE 312 AX, CROS PURE 312 AX, STYLETTO AX, CROS STYLETTO AX
MINIRECEIVER 3.0 45 dB SIZE 2 LEFT EAR	MINIRECEIVER 3.0 45 dB SIZE 2 LEFT EAR	10992554	1	\$51.36	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX, PURE 312 AX, CROS PURE 312 AX, STYLETTO AX, CROS STYLETTO AX
MINIRECEIVER 3.0 45 dB SIZE 3 RIGHT EAR	MINIRECEIVER 3.0 45 dB SIZE 3 RIGHT EAR	10992555	1	\$51.36	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX, PURE 312 AX, CROS PURE 312 AX, STYLETTO AX, CROS STYLETTO AX
MINIRECEIVER 3.0 45 dB SIZE 3 LEFT EAR	MINIRECEIVER 3.0 45 dB SIZE 3 LEFT EAR	10992556	1	\$51.36	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX, PURE 312 AX, CROS PURE 312 AX, STYLETTO AX, CROS STYLETTO AX
MINIRECEIVER 3.0 45 dB SIZE 4 RIGHT EAR	MINIRECEIVER 3.0 45 dB SIZE 4 RIGHT EAR	10992557	1	\$51.36	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX, PURE 312 AX, CROS PURE 312 AX, STYLETTO AX, CROS STYLETTO AX
MINIRECEIVER 3.0 45 dB SIZE 4 LEFT EAR	MINIRECEIVER 3.0 45 dB SIZE 4 LEFT EAR	10992558	1	\$51.36	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX, PURE 312 AX, CROS PURE 312 AX, STYLETTO AX, CROS STYLETTO AX

ROES codes (continued)

VA ROES ITEM	DESCRIPTION	ITEM #	QTY	PRICE	COMPATIBLE PRODUCTS
MINIRECEIVER 3.0 45 DB SIZE 5 RIGHT EAR	MINIRECEIVER 3.0 45 DB SIZE 5 RIGHT EAR	2102307	1	\$51.36	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX, PURE 312 AX, CROS PURE 312 AX, STYLETTO AX, CROS STYLETTO AX
MINIRECEIVER 3.0 45 DB SIZE 5 LEFT EAR	MINIRECEIVER 3.0 45 DB SIZE 5 LEFT EAR	21023211	1	\$51.36	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX, PURE 312 AX, CROS PURE 312 AX, STYLETTO AX, CROS STYLETTO AX
MINIRECEIVER 3.0 60 dB SIZE 0 RIGHT EAR	MINIRECEIVER 3.0 60 dB SIZE 0 RIGHT EAR	10992559	1	\$51.36	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX, PURE 312 AX, CROS PURE 312 AX, STYLETTO AX, CROS STYLETTO AX
MINIRECEIVER 3.0 60 dB SIZE 0 LEFT EAR	MINIRECEIVER 3.0 60 dB SIZE 0 LEFT EAR	10992560	1	\$51.36	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX, PURE 312 AX, CROS PURE 312 AX, STYLETTO AX, CROS STYLETTO AX
MINIRECEIVER 3.0 60 dB SIZE 1 RIGHT EAR	MINIRECEIVER 3.0 60 dB SIZE 1 RIGHT EAR	10992561	1	\$51.36	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX, PURE 312 AX, CROS PURE 312 AX, STYLETTO AX, CROS STYLETTO AX
MINIRECEIVER 3.0 60 dB SIZE 1 LEFT EAR	MINIRECEIVER 3.0 60 dB SIZE 1 LEFT EAR	10992562	1	\$51.36	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX, PURE 312 AX, CROS PURE 312 AX, STYLETTO AX, CROS STYLETTO AX
MINIRECEIVER 3.0 60 dB SIZE 2 RIGHT EAR	MINIRECEIVER 3.0 60 dB SIZE 2 RIGHT EAR	10992563	1	\$51.36	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX, PURE 312 AX, CROS PURE 312 AX, STYLETTO AX, CROS STYLETTO AX
MINIRECEIVER 3.0 60 dB SIZE 2 LEFT EAR	MINIRECEIVER 3.0 60 dB SIZE 2 LEFT EAR	10992564	1	\$51.36	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX, PURE 312 AX, CROS PURE 312 AX, STYLETTO AX, CROS STYLETTO AX
MINIRECEIVER 3.0 60 dB SIZE 3 RIGHT EAR	MINIRECEIVER 3.0 60 dB SIZE 3 RIGHT EAR	10992565	1	\$51.36	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX, PURE 312 AX, CROS PURE 312 AX, STYLETTO AX, CROS STYLETTO AX
MINIRECEIVER 3.0 60 dB SIZE 3 LEFT EAR	MINIRECEIVER 3.0 60 dB SIZE 3 LEFT EAR	10992566	1	\$51.36	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX, PURE 312 AX, CROS PURE 312 AX, STYLETTO AX, CROS STYLETTO AX
MINIRECEIVER 3.0 60 dB SIZE 4 RIGHT EAR	MINIRECEIVER 3.0 60 dB SIZE 4 RIGHT EAR	10992567	1	\$51.36	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX, PURE 312 AX, CROS PURE 312 AX, STYLETTO AX, CROS STYLETTO AX
MINIRECEIVER 3.0 60 dB SIZE 4 LEFT EAR	MINIRECEIVER 3.0 60 dB SIZE 4 LEFT EAR	10992568	1	\$51.36	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX, PURE 312 AX, CROS PURE 312 AX, STYLETTO AX, CROS STYLETTO AX

ROES codes (continued)

VA ROES ITEM	DESCRIPTION	ITEM #	QTY	PRICE	COMPATIBLE PRODUCTS
MINIRECEIVER 3.0 60 dB SIZE 5 RIGHT EAR	MINIRECEIVER 3.0 60 dB SIZE 5 RIGHT EAR	21023215	1	\$51.36	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX, PURE 312 AX, CROS PURE 312 AX, STYLETTO AX, CROS STYLETTO AX
MINIRECEIVER 3.0 60 dB SIZE 5 LEFT EAR	MINIRECEIVER 3.0 60 dB SIZE 5 LEFT EAR	2102316	1	\$51.36	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX, PURE 312 AX, CROS PURE 312 AX, STYLETTO AX, CROS STYLETTO AX
MINIRECEIVER 3.0 65 dB SIZE 0 RIGHT EAR	MINIRECEIVER 3.0 65 dB SIZE 0 RIGHT EAR	10992569	1	\$51.36	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX, PURE 312 AX, CROS PURE 312 AX, STYLETTO AX, CROS STYLETTO AX
MINIRECEIVER 3.0 65 dB SIZE 0 LEFT EAR	MINIRECEIVER 3.0 65 dB SIZE 0 LEFT EAR	10992570	1	\$51.36	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX, PURE 312 AX, CROS PURE 312 AX, STYLETTO AX, CROS STYLETTO AX
MINIRECEIVER 3.0 65 dB SIZE 1 RIGHT EAR	MINIRECEIVER 3.0 65 dB SIZE 1 RIGHT EAR	10992571	1	\$51.36	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX, PURE 312 AX, CROS PURE 312 AX, STYLETTO AX, CROS STYLETTO AX
MINIRECEIVER 3.0 65 dB SIZE 1 LEFT EAR	MINIRECEIVER 3.0 65 dB SIZE 1 LEFT EAR	10992572	1	\$51.36	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX, PURE 312 AX, CROS PURE 312 AX, STYLETTO AX, CROS STYLETTO AX
MINIRECEIVER 3.0 65 dB SIZE 2 RIGHT EAR	MINIRECEIVER 3.0 65 dB SIZE 2 RIGHT EAR	10992573	1	\$51.36	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX, PURE 312 AX, CROS PURE 312 AX, STYLETTO AX, CROS STYLETTO AX
MINIRECEIVER 3.0 65 dB SIZE 2 LEFT EAR	MINIRECEIVER 3.0 65 dB SIZE 2 LEFT EAR	10992574	1	\$51.36	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX, PURE 312 AX, CROS PURE 312 AX, STYLETTO AX, CROS STYLETTO AX
MINIRECEIVER 3.0 65 dB SIZE 3 RIGHT EAR	MINIRECEIVER 3.0 65 dB SIZE 3 RIGHT EAR	10992575	1	\$51.36	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX, PURE 312 AX, CROS PURE 312 AX, STYLETTO AX, CROS STYLETTO AX
MINIRECEIVER 3.0 65 dB SIZE 3 LEFT EAR	MINIRECEIVER 3.0 65 dB SIZE 3 LEFT EAR	10992576	1	\$51.36	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX, PURE 312 AX, CROS PURE 312 AX, STYLETTO AX, CROS STYLETTO AX
MINIRECEIVER 3.0 65 dB SIZE 4 RIGHT EAR	MINIRECEIVER 3.0 65 dB SIZE 4 RIGHT EAR	10992577	1	\$51.36	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX, PURE 312 AX, CROS PURE 312 AX, STYLETTO AX, CROS STYLETTO AX
MINIRECEIVER 3.0 65 dB SIZE 4 LEFT EAR	MINIRECEIVER 3.0 65 dB SIZE 4 LEFT EAR	10992578	1	\$51.36	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX, PURE 312 AX, CROS PURE 312 AX, STYLETTO AX, CROS STYLETTO AX

ROES codes (continued)

VA ROES ITEM	DESCRIPTION	ITEM #	QTY	PRICE	COMPATIBLE PRODUCTS
MINIRECEIVER 3.0 65 dB SIZE 5 RIGHT EAR	MINIRECEIVER 3.0 65 dB SIZE 5 RIGHT EAR	21023217	1	\$51.36	PURE CNG IX, CROS CNG IX, PURE CNG X RIC, PURE 312T X RIC, STYLETTO X RIC, PURE CNG NX RIC, PURE 312T NX RIC, PURE 13 NX RIC
MINIRECEIVER 3.0 65 dB SIZE 5 LEFT EAR	MINIRECEIVER 3.0 65 dB SIZE 5 LEFT EAR	21023218	1	\$51.36	PURE CNG IX, CROS CNG IX, PURE CNG X RIC, PURE 312T X RIC, STYLETTO X RIC, PURE CNG NX RIC, PURE 312T NX RIC, PURE 13 NX RIC
MINIRECEIVER COMPLETE SET	MINI RECEIVER 2.0 KIT	10960146	1	\$546.00	PURE CNG IX, CROS CNG IX, PURE CNG X RIC, PURE 312T X RIC, STYLETTO X RIC, PURE CNG NX RIC, PURE 312T NX RIC, PURE 13 NX RIC
S-RECEIVER 2.0 SIZE 0 L	MINI RECEIVER 2.0 45 dB SIZE 0 LEFT EAR	10669907	1	\$51.36	PURE CNG X RIC, PURE 312T X RIC, STYLETTO X RIC, PURE CNG NX RIC, PURE 312T NX RIC, PURE 13 NX RIC
S-RECEIVER 2.0 SIZE 0 R	MINI RECEIVER 2.0 45 dB SIZE 0 RIGHT EAR	10669908	1	\$51.36	PURE CNG X RIC, PURE 312T X RIC, STYLETTO X RIC, PURE CNG NX RIC, PURE 312T NX RIC, PURE 13 NX RIC
S-RECEIVER 2.0 SIZE 1 L	MINI RECEIVER 2.0 45 dB SIZE 1 LEFT EAR	10669909	1	\$51.36	PURE CNG X RIC, PURE 312T X RIC, STYLETTO X RIC, PURE CNG NX RIC, PURE 312T NX RIC, PURE 13 NX RIC
S-RECEIVER 2.0 SIZE 1 R	MINI RECEIVER 2.0 45 dB SIZE 1 RIGHT EAR	10669910	1	\$51.36	PURE CNG X RIC, PURE 312T X RIC, STYLETTO X RIC, PURE CNG NX RIC, PURE 312T NX RIC, PURE 13 NX RIC
S-RECEIVER 2.0 SIZE 2 L	MINI RECEIVER 2.0 45 dB SIZE 2 LEFT EAR	10669911	1	\$51.36	PURE CNG X RIC, PURE 312T X RIC, STYLETTO X RIC, PURE CNG NX RIC, PURE 312T NX RIC, PURE 13 NX RIC
S-RECEIVER 2.0 SIZE 2 R	MINI RECEIVER 2.0 45 dB SIZE 2 RIGHT EAR	10669912	1	\$51.36	PURE CNG X RIC, PURE 312T X RIC, STYLETTO X RIC, PURE CNG NX RIC, PURE 312T NX RIC, PURE 13 NX RIC
S-RECEIVER 2.0 SIZE 3 L	MINI RECEIVER 2.0 45 dB SIZE 3 LEFT EAR	10669913	1	\$51.36	PURE CNG X RIC, PURE 312T X RIC, STYLETTO X RIC, PURE CNG NX RIC, PURE 312T NX RIC, PURE 13 NX RIC
S-RECEIVER 2.0 SIZE 3 R	MINI RECEIVER 2.0 45 dB SIZE 3 RIGHT EAR	10669914	1	\$51.36	PURE CNG X RIC, PURE 312T X RIC, STYLETTO X RIC, PURE CNG NX RIC, PURE 312T NX RIC, PURE 13 NX RIC
M-RECEIVER 2.0 SIZE 0 L	MINI RECEIVER 2.0 60 dB SIZE 0 LEFT EAR	10824820	1	\$51.36	PURE CNG X RIC, PURE 312T X RIC, STYLETTO X RIC, PURE CNG NX RIC, PURE 312T NX RIC, PURE 13 NX RIC

ROES codes (continued)

VA ROES ITEM	DESCRIPTION	ITEM #	QTY	PRICE	COMPATIBLE PRODUCTS
M-RECEIVER 2.0 SIZE 0 R	MINI RECEIVER 2.0 60 dB SIZE 0 RIGHT EAR	10824821	1	\$51.36	PURE CNG X RIC, PURE 312T X RIC, STYLETTO X RIC, PURE CNG NX RIC, PURE 312T NX RIC, PURE 13 NX RIC
M-RECEIVER 2.0 SIZE 1 L	MINI RECEIVER 2.0 60 dB SIZE 1 LEFT EAR	10824822	1	\$51.36	PURE CNG X RIC, PURE 312T X RIC, STYLETTO X RIC, PURE CNG NX RIC, PURE 312T NX RIC, PURE 13 NX RIC
M-RECEIVER 2.0 SIZE 1 R	MINI RECEIVER 2.0 60 dB SIZE 1 RIGHT EAR	10824823	1	\$51.36	PURE CNG X RIC, PURE 312T X RIC, STYLETTO X RIC, PURE CNG NX RIC, PURE 312T NX RIC, PURE 13 NX RIC
M-RECEIVER 2.0 SIZE 2 L	MINI RECEIVER 2.0 60 dB SIZE 2 LEFT EAR	10824824	1	\$51.36	PURE CNG X RIC, PURE 312T X RIC, STYLETTO X RIC, PURE CNG NX RIC, PURE 312T NX RIC, PURE 13 NX RIC
M-RECEIVER 2.0 SIZE 2 R	MINI RECEIVER 2.0 60 dB SIZE 2 RIGHT EAR	10824825	1	\$51.36	PURE CNG X RIC, PURE 312T X RIC, STYLETTO X RIC, PURE CNG NX RIC, PURE 312T NX RIC, PURE 13 NX RIC
M-RECEIVER 2.0 SIZE 3 L	MINI RECEIVER 2.0 60 dB SIZE 3 LEFT EAR	10824826	1	\$51.36	PURE CNG X RIC, PURE 312T X RIC, STYLETTO X RIC, PURE CNG NX RIC, PURE 312T NX RIC, PURE 13 NX RIC
M-RECEIVER 2.0 SIZE 3 R	MINI RECEIVER 2.0 60dB SIZE 3 RIGHT EAR	10824827	1	\$51.36	PURE CNG X RIC, PURE 312T X RIC, STYLETTO X RIC, PURE CNG NX RIC, PURE 312T NX RIC, PURE 13 NX RIC
M-RECEIVER 2.0 SIZE 4 L	MINI RECEIVER 2.0 60 dB SIZE 1 LEFT EAR	10933054	1	\$51.36	PURE CNG X RIC, PURE 312T X RIC, STYLETTO X RIC, PURE CNG NX RIC, PURE 312T NX RIC, PURE 13 NX RIC
M-RECEIVER 2.0 SIZE 4 R	MINI RECEIVER 2.0 60 dB SIZE 1 RIGHT EAR	10933053	1	\$51.36	PURE CNG X RIC, PURE 312T X RIC, STYLETTO X RIC, PURE CNG NX RIC, PURE 312T NX RIC, PURE 13 NX RIC
P-RECEIVER 2.0 SIZE 0 L	MINI RECEIVER 2.0 70 dB SIZE 0 LEFT EAR	10669923	1	\$51.36	PURE CNG X RIC, PURE 312T X RIC, STYLETTO X RIC, PURE CNG NX RIC, PURE 312T NX RIC, PURE 13 NX RIC
P-RECEIVER 2.0 SIZE 0 R	MINI RECEIVER 2.0 70 dB SIZE 0 RIGHT EAR	10669924	1	\$51.36	PURE CNG X RIC, PURE 312T X RIC, STYLETTO X RIC, PURE CNG NX RIC, PURE 312T NX RIC, PURE 13 NX RIC
P-RECEIVER 2.0 SIZE 1 L	MINI RECEIVER 2.0 70 dB SIZE 1 LEFT EAR	10669925	1	\$51.36	PURE CNG X RIC, PURE 312T X RIC, STYLETTO X RIC, PURE CNG NX RIC, PURE 312T NX RIC, PURE 13 NX RIC

ROES codes (continued)

VA ROES ITEM	DESCRIPTION	ITEM #	QTY	PRICE	COMPATIBLE PRODUCTS
P-RECEIVER 2.0 SIZE 1 R	MINI RECEIVER 2.0 70 dB SIZE 1 RIGHT EAR	10669926	1	\$51.36	PURE CNG X RIC, PURE 312T X RIC, STYLETTO X RIC, PURE CNG NX RIC, PURE 312T NX RIC, PURE 13 NX RIC
P-RECEIVER 2.0 SIZE 2 L	MINI RECEIVER 2.0 70 dB SIZE 2 LEFT EAR	10669927	1	\$51.36	PURE CNG X RIC, PURE 312T X RIC, STYLETTO X RIC, PURE CNG NX RIC, PURE 312T NX RIC, PURE 13 NX RIC
P-RECEIVER 2.0 SIZE 2 R	MINI RECEIVER 2.0 70 dB SIZE 2 RIGHT EAR	10669928	1	\$51.36	PURE CNG X RIC, PURE 312T X RIC, STYLETTO X RIC, PURE CNG NX RIC, PURE 312T NX RIC, PURE 13 NX RIC
P-RECEIVER 2.0 SIZE 3 L	MINI RECEIVER 2.0 70 dB SIZE 3 LEFT EAR	10669929	1	\$51.36	PURE CNG X RIC, PURE 312T X RIC, STYLETTO X RIC, PURE CNG NX RIC, PURE 312T NX RIC, PURE 13 NX RIC
P-RECEIVER 2.0 SIZE 3 R	MINI RECEIVER 2.0 70 dB SIZE 3 RIGHT EAR	10669930	1	\$51.36	PURE CNG X RIC, PURE 312T X RIC, STYLETTO X RIC, PURE CNG NX RIC, PURE 312T NX RIC, PURE 13 NX RIC
THIN TUBES					
3.0 THIN TUBE KIT	THIN TUBE 3.0 FITTING KIT	10992763	1	\$93.60	MOTION CNG X BTE , MOTION CNG P X BTE
BINAURAL THINTUBE 3.0 S0	THIN TUBE 3.0 SIZE 0	10991609	2	\$8.53	MOTION CNG X BTE , MOTION CNG P X BTE
BINAURAL THINTUBE 3.0 S1	THIN TUBE 3.0 SIZE 1	10991610	2	\$8.53	MOTION CNG X BTE , MOTION CNG P X BTE
BINAURAL THINTUBE 3.0 S2	THIN TUBE 3.0 SIZE 2	10991611	2	\$8.53	MOTION CNG X BTE , MOTION CNG P X BTE
BINAURAL THINTUBE 3.0 S3	THIN TUBE 3.0 SIZE 3	10991612	2	\$8.53	MOTION CNG X BTE , MOTION CNG P X BTE
BINAURAL THINTUBE 3.0 S4	THIN TUBE 3.0 SIZE 4	10991613	2	\$8.53	MOTION CNG X BTE , MOTION CNG P X BTE
BINAURAL THINTUBE 3.0 S0 POWER	POWER THIN TUBE 3.0 SIZE 0	10992492	2	\$8.53	MOTION CNG X BTE , MOTION CNG P X BTE
BINAURAL THINTUBE 3.0 S1 POWER	POWER THIN TUBE 3.0 SIZE 1	10992493	2	\$8.53	MOTION CNG X BTE , MOTION CNG P X BTE
BINAURAL THINTUBE 3.0 S2 POWER	POWER THIN TUBE 3.0 SIZE 2	10992494	2	\$8.53	MOTION CNG X BTE , MOTION CNG P X BTE

ROES codes (continued)

VA ROES ITEM	DESCRIPTION	ITEM #	QTY	PRICE	COMPATIBLE PRODUCTS
BINAURAL THINTUBE 3.0 S3 POWER	POWER THIN TUBE 3.0 SIZE 3	10992495	2	\$8.53	MOTION CNG X BTE , MOTION CNG P X BTE
BINAURAL THINTUBE 3.0 S4 POWER	POWER THIN TUBE 3.0 SIZE 4	10992496	2	\$8.53	MOTION CNG X BTE , MOTION CNG P X BTE
TUBE THIN 2.0 FITTING SET	THIN TUBE 2.0 FITTING KIT	10828299	1	\$169.71	MOTION CNG NX BTE, MOTION 13 NX BTE, MOTION 13P NX BTE
THIN TUBE 2.0 1L	THIN TUBE 2.0 SIZE 1 LEFT	10828291	5	\$8.53	MOTION CNG NX BTE, MOTION 13 NX BTE, MOTION 13P NX BTE
THIN TUBE 2.0 1R	THIN TUBE 2.0 SIZE 1 RIGHT	10828292	5	\$8.53	MOTION CNG NX BTE, MOTION 13 NX BTE, MOTION 13P NX BTE
THIN TUBE 2.0 2L	THIN TUBE 2.0 SIZE 2 LEFT	10828293	5	\$8.53	MOTION CNG NX BTE, MOTION 13 NX BTE, MOTION 13P NX BTE
THIN TUBE 2.0 2R	THIN TUBE 2.0 SIZE 2 RIGHT	10828294	5	\$8.53	MOTION CNG NX BTE, MOTION 13 NX BTE, MOTION 13P NX BTE
THIN TUBE 2.0 3L	THIN TUBE 2.0 SIZE 3 LEFT	10828295	5	\$8.53	MOTION CNG NX BTE, MOTION 13 NX BTE, MOTION 13P NX BTE
THIN TUBE 2.0 3R	THIN TUBE 2.0 SIZE 3 RIGHT	10828296	5	\$8.53	MOTION CNG NX BTE, MOTION 13 NX BTE, MOTION 13P NX BTE
THIN TUBE 2.0 4L	THIN TUBE 2.0 SIZE 4 LEFT	10828297	5	\$8.53	MOTION CNG NX BTE, MOTION 13 NX BTE, MOTION 13P NX BTE
THIN TUBE 2.0 4R	THIN TUBE 2.0 SIZE 4 RIGHT	10828298	5	\$8.53	MOTION CNG NX BTE, MOTION 13 NX BTE, MOTION 13P NX BTE
DOMES AND SLEEVES					
EARTIP 3.0 5MM OPEN	EARTIP 3.0 S OPEN	10993678	6	\$8.53	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX, MOTION CNG X BTE, MOTION CNG P BTE, Pure 312 AX, CROS 312 AX, Styletto AX, CROS Styletto AX
EARTIP 3.0 7MM OPEN	EARTIP 3.0 M OPEN	10993680	6	\$8.53	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX, MOTION CNG X BTE, MOTION CNG P BTE, Pure 312 AX, CROS 312 AX, Styletto AX, CROS Styletto AX
EARTIP 3.0 10MM OPEN	EARTIP 3.0 L OPEN	10993681	6	\$8.53	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX, MOTION CNG X BTE, MOTION CNG P BTE, Pure 312 AX, CROS 312 AX, Styletto AX, CROS Styletto AX

ROES codes (continued)

VA ROES ITEM	DESCRIPTION	ITEM #	QTY	PRICE	COMPATIBLE PRODUCTS
EARTIP 3.0 8MM TULIP	EARTIP 3.0 M TULIP	10993682	6	\$8.53	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX, MOTION CNG X BTE, MOTION CNG P BTE, Pure 312 AX, CROS 312 AX, Styletto AX, CROS Styletto AX
EARTIP 3.0 12MM TULIP	EARTIP 3.0 L TULIP	10993683	6	\$8.53	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX, MOTION CNG X BTE, MOTION CNG P BTE, Pure 312 AX, CROS 312 AX, Styletto AX, CROS Styletto AX
EARTIP 3.0 S ROUND DOME 1	EARTIP 3.0 S ROUND DOME 1	21022728	6	\$8.53	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX, MOTION CNG X BTE, MOTION CNG P BTE, Pure 312 AX, CROS 312 AX, Styletto AX, CROS Styletto AX
EARTIP 3.0 M ROUND DOME 1	EARTIP 3.0 M ROUND DOME 1	21022729	6	\$8.53	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX, MOTION CNG X BTE, MOTION CNG P BTE, Pure 312 AX, CROS 312 AX, Styletto AX, CROS Styletto AX
EARTIP 3.0 L ROUND DOME 1	EARTIP 3.0 L ROUND DOME 1	21022730	6	\$8.53	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX, MOTION CNG X BTE, MOTION CNG P BTE, Pure 312 AX, CROS 312 AX, Styletto AX, CROS Styletto AX
SLEEVE 3.0 VENTED XS	SLEEVE 3.0 XS VENTED	10992863	6	\$8.53	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX, MOTION CNG X BTE, MOTION CNG P BTE, Pure 312 AX, CROS 312 AX, Styletto AX, CROS Styletto AX
SLEEVE 3.0 VENTED S	SLEEVE 3.0 VENTED S TBLK	10992864	6	\$8.53	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX, MOTION CNG X BTE, MOTION CNG P BTE, Pure 312 AX, CROS 312 AX, Styletto AX, CROS Styletto AX
SLEEVE 3.0 VENTED M	SLEEVE 3.0 VENTED M TBLK	10992865	6	\$8.53	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX, MOTION CNG X BTE, MOTION CNG P BTE, Pure 312 AX, CROS 312 AX, Styletto AX, CROS Styletto AX
SLEEVE 3.0 VENTED L	SLEEVE 3.0 VENTED L TBLK	10992866	6	\$8.53	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX, MOTION CNG X BTE, MOTION CNG P BTE, Pure 312 AX, CROS 312 AX, Styletto AX, CROS Styletto AX

ROES codes (continued)

VA ROES ITEM	DESCRIPTION	ITEM #	QTY	PRICE	COMPATIBLE PRODUCTS
SLEEVE 3.0 CLOSED XS	SLEEVE 3.0 CLOSED XS TBLK	10992867	6	\$8.53	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX, MOTION CNG X BTE, MOTION CNG P BTE, Pure 312 AX, CROS 312 AX, Styletto AX, CROS Styletto AX
SLEEVE 3.0 CLOSED S	SLEEVE 3.0 CLOSED S TBLK	10992868	6	\$8.53	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX, MOTION CNG X BTE, MOTION CNG P BTE, Pure 312 AX, CROS 312 AX, Styletto AX, CROS Styletto AX
SLEEVE 3.0 CLOSED M	SLEEVE 3.0 CLOSED M TBLK	10992869	6	\$8.53	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX, MOTION CNG X BTE, MOTION CNG P BTE, Pure 312 AX, CROS 312 AX, Styletto AX, CROS Styletto AX
SLEEVE 3.0 CLOSED L	SLEEVE 3.0 CLOSED L TBLK	10992870	6	\$8.53	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX, MOTION CNG X BTE, MOTION CNG P BTE, Pure 312 AX, CROS 312 AX, Styletto AX, CROS Styletto AX
SLEEVE 3.0 POWER XS	SLEEVE 3.0 POWER XS CXX	10997192	6	\$8.53	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX, MOTION CNG X BTE, MOTION CNG P BTE, Pure 312 AX, CROS 312 AX, Styletto AX, CROS Styletto AX
SLEEVE 3.0 POWER S	SLEEVE 3.0 POWER S CXX	10997193	6	\$8.53	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX, MOTION CNG X BTE, MOTION CNG P BTE, Pure 312 AX, CROS 312 AX, Styletto AX, CROS Styletto AX
SLEEVE 3.0 POWER M	SLEEVE 3.0 POWER M CXX	10997194	6	\$8.53	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX, MOTION CNG X BTE, MOTION CNG P BTE, Pure 312 AX, CROS 312 AX, Styletto AX, CROS Styletto AX
SLEEVE 3.0 POWER L	SLEEVE 3.0 POWER L CXX	10997195	6	\$8.53	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX, MOTION CNG X BTE, MOTION CNG P BTE, Pure 312 AX, CROS 312 AX, Styletto AX, CROS Styletto AX
CLICK SLEEVES XS VENTED	CLICK SLEEVE 2.0 XS VENTED	10951152	6	\$8.53	PURE CNG X RIC, PURE 312T X RIC, STYLETTO X RIC, PURE CNG NX RIC, PURE 312T NX RIC, PURE 13 NX RIC

ROES codes (continued)

VA ROES ITEM	DESCRIPTION	ITEM #	QTY	PRICE	COMPATIBLE PRODUCTS
CLICK SLEEVES S VENTED	CLICK SLEEVE 2.0 S VENTED	10951146	6	\$8.53	PURE CNG X RIC, PURE 312T X RIC, STYLETTO X RIC, PURE CNG NX RIC, PURE 312T NX RIC, PURE 13 NX RIC
CLICK SLEEVES M VENTED	CLICK SLEEVE 2.0 M VENTED	10951148	6	\$8.53	PURE CNG X RIC, PURE 312T X RIC, STYLETTO X RIC, PURE CNG NX RIC, PURE 312T NX RIC, PURE 13 NX RIC
CLICK SLEEVES L VENTED	CLICK SLEEVE 2.0 L VENTED	10951150	6	\$8.53	PURE CNG X RIC, PURE 312T X RIC, STYLETTO X RIC, PURE CNG NX RIC, PURE 312T NX RIC, PURE 13 NX RIC
CLICK SLEEVES S CLOSED	CLICK SLEEVE 2.0 S CLOSED	10951147	6	\$8.53	PURE CNG X RIC, PURE 312T X RIC, STYLETTO X RIC, PURE CNG NX RIC, PURE 312T NX RIC, PURE 13 NX RIC
CLICK SLEEVES M CLOSED	CLICK SLEEVE 2.0 M CLOSED	10951149	6	\$8.53	PURE CNG X RIC, PURE 312T X RIC, STYLETTO X RIC, PURE CNG NX RIC, PURE 312T NX RIC, PURE 13 NX RIC
CLICK SLEEVES L CLOSED	CLICK SLEEVE 2.0 L CLOSED	10951151	6	\$8.53	PURE CNG X RIC, PURE 312T X RIC, STYLETTO X RIC, PURE CNG NX RIC, PURE 312T NX RIC, PURE 13 NX RIC
CLICK DOMES OPEN 4MM	CLICK DOME S OPEN	10426016	6	\$5.12	PURE CNG X RIC, PURE 312T X RIC, STYLETTO X RIC, PURE CNG NX RIC, PURE 312T NX RIC, PURE 13 NX RIC
CLICK DOMES OPEN 8MM	CLICK DOME M OPEN	10426014	6	\$5.12	PURE CNG X RIC, PURE 312T X RIC, STYLETTO X RIC, PURE CNG NX RIC, PURE 312T NX RIC, PURE 13 NX RIC
CLICK DOMES OPEN 10MM	CLICK DOME L OPEN	10426015	6	\$5.12	PURE CNG X RIC, PURE 312T X RIC, STYLETTO X RIC, PURE CNG NX RIC, PURE 312T NX RIC, PURE 13 NX RIC
CLICK DOMES CLOSED 6MM	CLICK DOME S CLOSED	10426017	6	\$5.12	PURE CNG X RIC, PURE 312T X RIC, STYLETTO X RIC, PURE CNG NX RIC, PURE 312T NX RIC, PURE 13 NX RIC
CLICK DOMES CLOSED 8MM	CLICK DOME M CLOSED	10426012	6	\$5.12	PURE CNG X RIC, PURE 312T X RIC, STYLETTO X RIC, PURE CNG NX RIC, PURE 312T NX RIC, PURE 13 NX RIC
CLICK DOMES CLOSED 10MM	CLICK DOME L CLOSED	10426013	6	\$5.12	PURE CNG X RIC, PURE 312T X RIC, STYLETTO X RIC, PURE CNG NX RIC, PURE 312T NX RIC, PURE 13 NX RIC

ROES codes (continued)

VA ROES ITEM	DESCRIPTION	ITEM #	QTY	PRICE	COMPATIBLE PRODUCTS
CLICK DOMES DOUBLE 8/10MM	CLICK DOME S DOUBLE	10426018	6	\$5.12	PURE CNG X RIC, PURE 312T X RIC, STYLETTO X RIC, PURE CNG NX RIC, PURE 312T NX RIC, PURE 13 NX RIC
CLICK DOMES DOUBLE 10/12MM	CLICK DOME M DOUBLE	10426019	6	\$5.12	PURE CNG X RIC, PURE 312T X RIC, STYLETTO X RIC, PURE CNG NX RIC, PURE 312T NX RIC, PURE 13 NX RIC
DOMES 8/12MM SEMI OPEN CLICK RIC	CLICK DOME M SEMI OPEN	10670790	6	\$5.12	PURE CNG X RIC, PURE 312T X RIC, STYLETTO X RIC, PURE CNG NX RIC, PURE 312T NX RIC, PURE 13 NX RIC
LIFE TIP 8 MM OPEN	LIFE TIP REFILL 8MM	10054916	1	\$8.53	MOTION CNG NX BTE, MOTION 13 NX BTE. MOTION 13P NX BTE
LIFE TIP 8 MM CLOSED	LIFE TIP REFILL 8MM FULL	10169330	1	\$8.53	MOTION CNG NX BTE, MOTION 13 NX BTE. MOTION 13P NX BTE
LIFE TIP 4 MM	LIFE TIP REFILL 4 MM	10169328	1	\$8.53	MOTION CNG NX BTE, MOTION 13 NX BTE. MOTION 13P NX BTE
LIFE TIP 6 MM	LIFE TIP REFILL 6MM	10169329	1	\$8.53	MOTION CNG NX BTE, MOTION 13 NX BTE. MOTION 13P NX BTE
LIFE TIP 10 MM	LIFE TIP REFILL 10MM	10054917	1	\$8.53	MOTION CNG NX BTE, MOTION 13 NX BTE. MOTION 13P NX BTE
DOUBLE TIP 8/10	LIFE TIP DOUBLE REFILL 8/10MM	10175566	1	\$8.53	MOTION CNG NX BTE, MOTION 13 NX BTE. MOTION 13P NX BTE
DOUBLE TIP 10/12	LIFE TIP DOUBLE REFILL 10/12MM	10175715	1	\$8.53	MOTION CNG NX BTE, MOTION 13 NX BTE. MOTION 13P NX BTE
CHARGERS					
CLINIC MULTI-CHARGER		21026076	1	\$79	ALL HEARING AIDS THAT HAVE GALVANIC CHARGING
CLINIC MULTI-CHARGER STAND		21027726	1	\$15	ALL HEARING AIDS THAT HAVE GALVANIC CHARGING
INSIO CNG AX CHARGER	CHARGER FOR INSIO CUSTOM HEARING AIDS	10997503	1	\$175.76	INSIO CNG AX
PORTABLE CHARGER FOR PURE CNG AX	PORTABLE CHARGER FOR PURE C&G AX	10993118	1	\$175.76	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX

ROES codes (continued)

VA ROES ITEM	DESCRIPTION	ITEM #	QTY	PRICE	COMPATIBLE PRODUCTS
DRY & CLEAN CHARGER FOR PURE CNG AX	DRY&CLEAN CHARGER FOR PURE C&G AX	10991650	1	\$218.40	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX
STANDARD CHARGER FOR PURE CNG AX	CHARGER FOR PURE C&G AX	10996345	1	\$175.76	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX
CHARGER STYLETTO AX	CHARGER FOR STYLETTO AX	21003216	1	\$175.76	STYLETTO AX RIC, CROS STYLETTO AX
DRY&CLEAN CHARGER STYLETTO AX	DRY&CLEAN CHARGER FOR STYLETTO AX	21007593	1	\$218.40	STYLETTO AX, CROS STYLETTO AX
DRY & CLEAN CHARGER MOTION X BTE P	DRY & CLEAN CHARGER FOR MOTION CNG P X BTE	10991638	1	\$218.40	MOTION CNG X P BTE
DRY & CLEAN CHARGER MOTION X BTE SP	DRY & CLEAN CHARGER FOR MOTION CNG SP X BTE	10991644	1	\$218.40	MOTION CNG X SP BTE
CHARGER MOTION XBTE P	CHARGER FOR MOTION CNG P X BTE	10996335	1	\$175.76	MOTION CNG P BTE
CHARGER MOTION X BTE SP	CHARGER FOR MOTION CNG SP X BTE	10996340	1	\$175.76	MOTION CNG SP BTE
INDUCTIVE CHARGER II	INDUCTIVE CHARGER II FOR PURE AND MOTION	10966942	1	\$165.57	PURE CNG X RIC, MOTION CNG X, MOTION CNG NX
CHARGER STYLETTO X	CHARGER FOR STYLETTO X	10972015	1	\$175.76	STYLETTO X RIC
CHARGER ACTIVE PRO	CHARGER FOR ACTIVE PRO	10991347	1	\$175.76	ACTIVE PRO
CHARGER INDUCTIVE SET LI-ION	LI-ION INDUCTIVE CHARGER SET	10948199	1	\$175.76	CELLION PX, PURE CNG NX, MOTION CNG NX
WAXGUARDS					
WAXGUARD 3.0 NANOCARE	EARWEAR 3.0 NANOCARE WAXGUARD	10993607	1	\$0.57	PURE CNG AX, INSIO AX, MOTION CNG X BTE, MOTION CNG P X BTE, STYLETTO AX

ROES codes (continued)

VA ROES ITEM	DESCRIPTION	ITEM #	QTY	PRICE	COMPATIBLE PRODUCTS
CERUSTOP XL	WAXGUARD FOR HP EAR-MOLD	10999334	1	\$0.00	PURE CNG AX HP MOLD
WAXGUARD RIC MINI	MINI RECEIVER WAXGUARD	10966257	10	\$2.98	PURE CNG X RIC, PURE 312T X RIC, STYLETTO X RIC, PURE CNG NX RIC, PURE 312T NX RIC, PURE 13 NX RIC
QUICKGUARD PACK	CXX QuickGuard	10962504	1	\$3.41	INSIO NX CUSTOMS
WAXGUARD C-GRID	C-GRID WAXGUARD	10423507	16	\$8.53	INSIO NX CUSTOMS
WAX SPRING,RCVR TUBE ITE	WAX SPRING FOR RCVR TUBE	07080208	1	\$0.00	INSIO NX CUSTOMS
CONCHA LOCKS					
CONCHA LOCK 3.0 SLEEVES S RECEIVERS		21024885	10	\$0.00	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX, PURE 312 AX, CROS 312 AX, STYLETTO AX, CROS STYLETTO AX, MOTION CNG X BTEs
CONCHA LOCK 3.0 SLEEVES M RECEIVERS		21024886	10	\$0.00	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX, PURE 312 AX, CROS 312 AX, STYLETTO AX, CROS STYLETTO AX, MOTION CNG X BTEs
CONCHA LOCK 3.0 SLEEVES P RECEIVERS		21024887	10	\$0.00	PURE CNG IX, CROS CNG IX, PURE CNG AX, CROS CNG AX, PURE 312 AX, CROS 312 AX, STYLETTO AX, CROS STYLETTO AX, MOTION CNG X BTEs
RECEIVER CONCHA LOCK S45dB	MINI RECEIVER 2.0 CONCHA LOCK FOR S RECEIVER	10601877	10	\$0.00	PURE CNG X RIC, PURE 312T X RIC, PURE CNG NX RIC, PURE 312T NX RIC, PURE 13 NX RIC
RECEIVER CONCHA LOCK M60dB	MINI RECEIVER 2.0 CONCHA LOCK FOR M RECEIVER	10937826	10	\$0.00	PURE CNG X RIC, PURE 312T X RIC, PURE CNG NX RIC, PURE 312T NX RIC, PURE 13 NX RIC
RECEIVER CONCHA LOCK P70db	MINI RECEIVER 2.0 CONCHA LOCK FOR P RECEIVER	10673516	10	\$0.00	PURE CNG X RIC, PURE 312T X RIC, PURE CNG NX RIC, PURE 312T NX RIC, PURE 13 NX RIC
EARHOOKS					

ROES codes (continued)

VA ROES ITEM	DESCRIPTION	ITEM #	QTY	PRICE	COMPATIBLE PRODUCTS
EARHOOK MOTION C-G NX	EARHOOK FOR MOTION CNG NX BTE	10959864	1	\$0.00	MOTION CNG NX BTE
MOTION PX/NX EARHOOK	EARHOOK FOR MOTION 13 NX BTE	10947168	1	\$0.00	MOTION 13 NX BTE
THINTUBE ADAPTER MOTION	THIN TUBE ADAPTER FOR MOTION CNG NX BTE	10958343	1	\$3.12	MOTION CNG NX BTE
MOTION PX/NX TUBE ADAPTER	THIN TUBE ADAPTER FOR MOTION 13 NX BTE	10947303	1	\$0.00	MOTION 13 NX BTE
EARHOOK MOTION X SMALL	EARHOOK MOTION X SMALL FOR	10980403	1	\$0.00	MOTION CNG X BTE, MOTION CNG P X BTE
EARHOOK MOTION X STANDARD	EARHOOK MOTION X STANDARD	10980402	1	\$0.00	MOTION CNG X BTE, MOTION CNG P X BTE
EARHOOK MOTION X SP SMALL	EARHOOK MOTION X SP SMALL	10993620	1	\$0.00	MOTION CNG SP X BTE
MIC COVERS					
CUSTOM MIC COVER BEIGE, TAN, MOCHA	MIC COVERS FOR CUSTOM HEARING AIDS - BEIGE, TAN, MOCHA	10947542	1	\$0.00	INSIO C&G AX, INSIO NX
CUSTOM MIC COVER BROWN	MIC COVER FOR CUSTOM HEARING AIDS - BROWN	10947776	1	\$0.00	INSIO C&G AX, INSIO NX
CUSTOM MIC COVER DARK BRN	MIC COVER FOR CUSTOM HEARING AIDS - DARK BROWN	21016202	1	\$0.00	INSIO C&G AX, INSIO NX
CUSTOM MIC COVER BLK	MIC COVER FOR CUSTOM HEARING AIDS - BLACK	21016200	1	\$0.00	INSIO C&G AX, INSIO NX

ROES codes (continued)

VA ROES ITEM	DESCRIPTION	ITEM #	QTY	PRICE	COMPATIBLE PRODUCTS
MISC TOOLS					
3.0 MEASURING TOOL	EARWEAR 3.0 MEASURING TOOL	10991403	1	\$0.00	PURE CNG AX, CROS CNG AX, MOTION CNG X BTE, MOTION CNG P X BTE
EAR-TIP 3.0 INSERTION TOOL	EARWEAR 3.0 INSERTION TOOL	10994214	1	\$0.00	PURE CNG AX, CROS CNG AX, MOTION CNG X BTE, MOTION CNG P X BTE
TOOL MEASURING	RECEIVER MEASURING TOOL	10174571	1	\$0.00	PURE CNG X RIC , PURE 312T X RIC, STYLETTO X RIC. PURE CNG NX RIC, PURE 312T NX RIC, PURE 13 NX RIC
MINI RECEIVER TOOL	MINI RECEIVER REMOVAL TOOL	10674370	1	\$0.00	PURE CNG X , PURE 312T X RIC, STYLETTTO X RIC, PURE CNG NX, PURE 312T NX RIC, PURE 13 NX RIC
THIN TUBE 3.0 PIN EJECTOR TOOL	EARHOOK REMOVAL TOOL	10600047	1	\$0.00	MOTION CNG X BTE, MOTION CNG P X BTE
TOOL CLICK MOLD	CLICK MOLD SERVICE TOOL	10597930	1	\$0.00	PURE CNG X RIC, PURE 312T X RIC, STYLETTO X RIC, PURE CNG NX RIC, PURE 312T NX RIC, PURE 13 NX RIC
EARTIP 3.0 HANDLING TOOL	3.0 EARMOLD SERVICE TOOL	10984033	1	\$0.00	MOTION CNG X BTE, MOTION CNG P X BTE
ACCESSORY PARTS					
STREAMLINE TV POWER SUPPLY	POWER SUPPLY FOR STREAM-LINE TV	10944815	1	\$0.00	STREAMLINE TV
SMARTCONNECT POWER ADAPT SET	SMARTCONNECT POWER ADAPT SET	10824346	1	\$0.00	SMARTCONNECT

The Bluetooth® word mark and logos are owned by the Bluetooth SIG, Inc. and any use of such marks by Sivantos is under license. Other trademarks and trade names are those of their respective owners.

Android is a trademark of Google Inc.

iOS is a trademark or registered trademark of Cisco in the U.S. and other countries and is used under license.

"Made for iPhone" means that an electronic accessory has been designed to connect specifically to iPhone and has been certified by the developer to meet Apple performance standards. Apple is not responsible for the operation of this device or its compliance with safety and regulatory standards. Please note that the use of this accessory with iPhone may affect wireless performance.



Our mission is to help improve the quality of life of Veterans,
Active Military and American Indians with better hearing.

Signia Government Services: (800) 955-4327 | Consumer Hotline: (800) 350-6093